


CardioPet ECG Device with iOS* and Android* Tablets

Quick Reference Guide



Record and submit a CardioPet ECG

1. Open the CardioPet* App.
2. Attach CardioPet* comfort clips to the patient.
3. Turn on the CardioPet* ECG device.
The wireless communication activates and the light on the device illuminates. The ECG begins streaming automatically. (If it does not, refer to the [CardioPet User Guide](#) for additional information.)
4. If prompted, select the patient species: Canine, Feline, or Other.
5. Click **Start Recording**.
Note: Recording stops automatically after 30 or 60 seconds (configurable in Settings).
6. Click **Select Patient & Save**.
7. Choose an existing patient, or add a new patient using the **Add** icon  and then click **Save** to confirm.
Note: If the CardioPet app is integrated with IDEXX-PACS* Imaging Software or IDEXX Web PACS* software, the ECG will be transferred to the patient's card in the software. Contact IDEXX Telemedicine Consultants at **1-800-726-1212** for more information about the integration workflow.
8. If the CardioPet app is not integrated with IDEXX-PACS or IDEXX Web PACS software, click **Start a Case** and then select **Finish Now** or **Finish Later**.
9. Complete the case submission on vetmedstat.com.

For additional help, visit idexx.com/telemedresources or call 1-800-726-1212.