

Introducing the redesigned Advanced Viewer

with more viewing room, plus configurable options

- Patient history is at the bottom of the window, leaving more room in the main viewing area.
- Management and information tools are easily accessible on the right.
- New Viewer Settings window lets you:
 - Customize the toolbar.
 - Customize mouse button shortcuts.

Wider viewing area

New Viewer Settings window with configurable options

Management and information tools always visible; no menus

Patient history at bottom

Previous viewer


Management and information tools hidden under menu

Patient history on right

Smaller viewing area


[Learn more →](#)

Wider viewing area

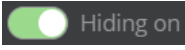- With patient history at the bottom, there's more room in the main viewing area.
- Click the **+** to expand a single study, scroll right/left to see more thumbnails, or expand the pane to see up to three rows of thumbnails.
- For even more space, toggle the hiding switch to **Hiding On**  in the toolbar to hide thumbnails.

Thumbnails at the bottom of the window


Click to expand study

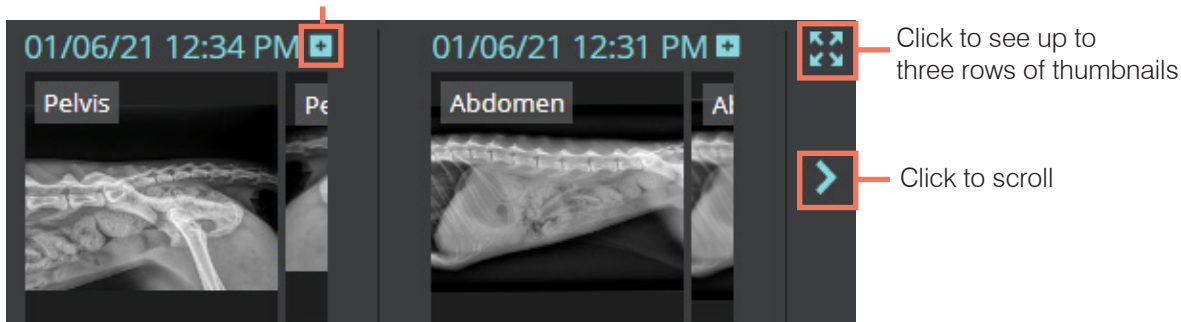

An expanded individual study




The expanded patient history pane with three rows of thumbnails

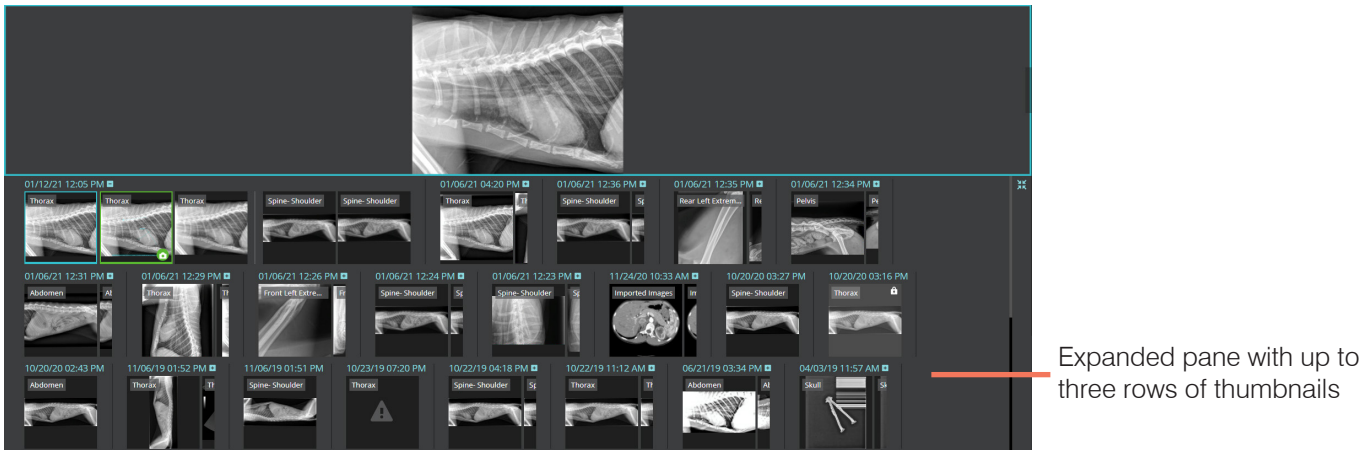

Management and information tools are always visible on the right


- New **action toolbar** on the right provides quick access to management and information tools.
- The More Options menu has been removed.
- New icons display file counts for the current study.

Viewer with action toolbar

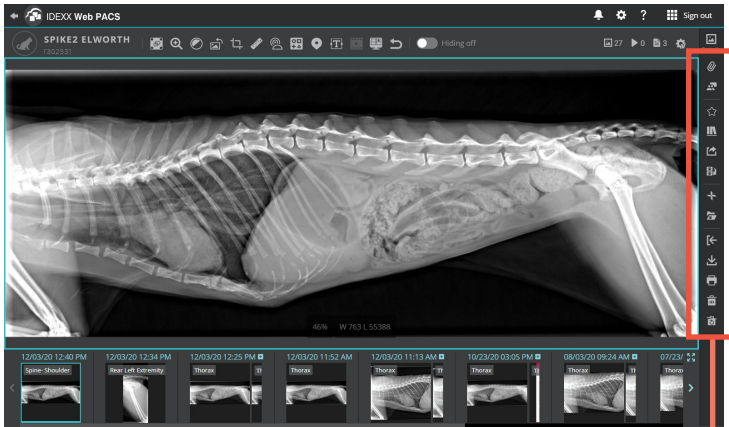

Action toolbar

File counts

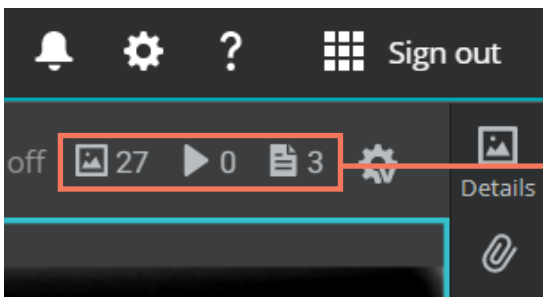



- Images
- Playable files
- PDFs

Action toolbar

Study details	—		Details
			Telemedicine report
Interactive collaboration	—		
			Make reference
Reference Image Library	—		
			Share
Reassign	—		
			View in VetConnect PLUS
Manage client/patient	—		
			Import
Download	—		
			Print
Delete	—		
			Restore deleted

New Viewer Settings to configure the toolbar, mouse, and more

Click  in the Advanced Viewer to open the new Viewer Settings window

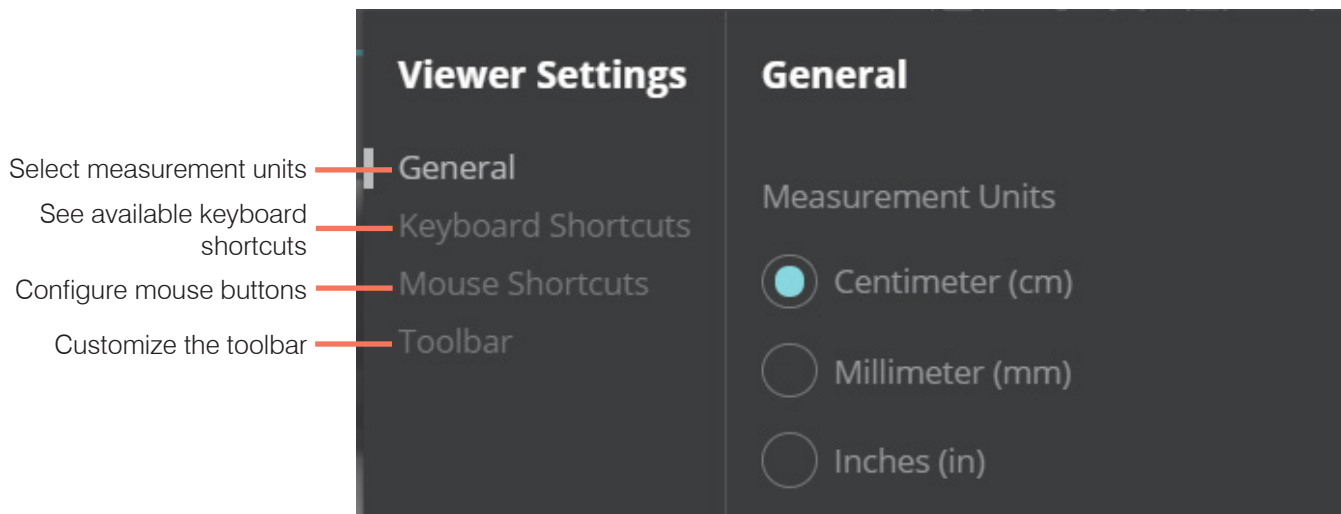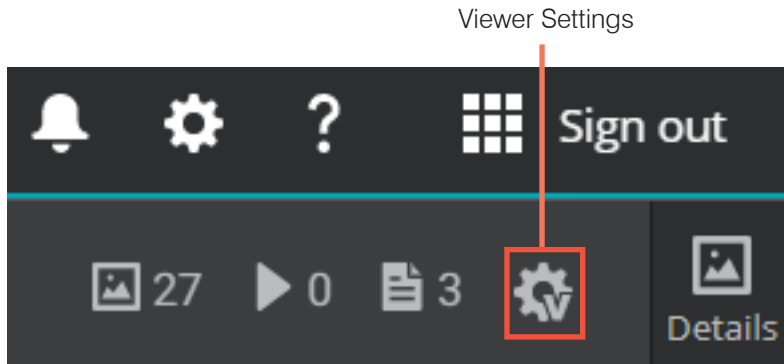

- Choose metric vs. nonmetric measurement units.
Note: This setting was moved from the general settings window.
- See available keyboard shortcuts.
- Customize mouse buttons to choose which button pans, zooms, or changes window/level when you hold down the button.
- Hide or show individual tools in the toolbar; change tool order.

For more information about the features and functions of the Advanced Viewer, click **Help ?** in the toolbar.