SNAP® Lepto.

In vitro diagnostic for the detection of anti-*Leptospira* antibodies in canine serum.

Precautions and warnings

- All wastes should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- This test kit cannot differentiate between antibodies to pathogenic and nonpathogenic serovars of *Leptospira*.
- This test kit cannot differentiate between antibodies produced as a result of a natural infection with the organism and those produced by vaccination.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.

Storage

- Store at 2-8°C until expiration date.
- All components must be at room temperature (18-25°C) before running the test.

Kit components

Item	Reagents	Quantity
1	1 bottle of <i>Leptospira</i> Antigen: HRPO Conjugate (Preserved with ProClin™ 150)	3.8 mL
2	SNAP Device	5 or 10
Reagents contained in each device:		
	1 Wash solution (preserved with ProClin™ 150)	0.4 mL
	Substrate solution	0.6 mL

Conjugate - H317/H412/P261/P280/P302 + P332/P333 + P313: May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mists/vapors. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

Wash Solution - H317/H318/H402/H412/P261/P280/P302 + P331 + P333/P337 + P313/P333+P313: May cause an allergic skin reaction. Causes serious eye irritation. Harmful to aquatic life. Harmful to aquatic life with long lasting effects. Avoid breathing mists/vapors. Wear eye protection/face protection. Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

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Sample information

- Samples must be at room temperature 18-25°C before beginning the test procedure.
- Serum, either fresh or stored at 2-8°C for up to one week, can be used.
- For longer storage, serum can be frozen (-20°C or colder) and then re centrifuged before use.
- Hemolyzed or lipemic samples will not affect test results.

Test procedure

- Allow all components to equilibrate at room temperature 18-25°C for 30 minutes before use. **Do not heat.**

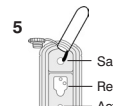
- Using the pipette provided, dispense **3 drops of sample** into a new sample tube.

- Holding the bottle vertical, add **4 drops of conjugate** to the sample tube.

- Cap the sample tube and mix it thoroughly by **inverting it 3-5 times**.

- Place the device on a horizontal surface. Add the entire contents of the sample tube to the sample well, being careful not to splash the contents outside of the sample well. The sample will flow across the result window, reaching the activation circle in 30-60 seconds. Some sample may remain in the sample well.

- When color **FIRST** appears in the activation circle, push the activator firmly until it is flush with the device body.

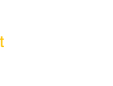
Note: Some samples may not flow to the activation circle within 60 seconds, and therefore, the circle may not turn color. In this case, press the activator after the sample has flowed across the result window.

- Read test result at **10 minutes**. **Note:** The positive control may develop sooner, but results are not complete until 10 minutes.

- Read test result at **10 minutes**. **Note:** The positive control may develop sooner, but results are not complete until 10 minutes.

Interpreting the test results

Positive result
Any color development in the sample spots indicates the presence of anti-*Leptospira* antibody in the sample.

Positive control → Sample spot

Negative result
Only positive control spot develops color.

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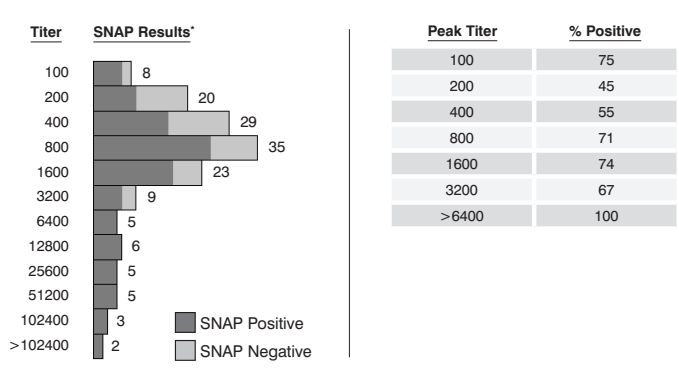
Invalid results
• **Background** — If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if colored background obscures test result, repeat the test.
• **No color development** — If the positive control does not develop color, repeat the test.

- Background** — If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if colored background obscures test result, repeat the test.
- No color development** — If the positive control does not develop color, repeat the test.

Performance summary by MAT Titer

*Numbers shown represent total MAT positive samples per titer.

Titer	SNAP Positive	SNAP Negative
100	8	0
200	20	0
400	29	0
800	35	0
1600	5	0
3200	9	0
6400	5	0
12800	3	0
25600	2	0
>102400	0	0

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© 2022 IDEXX Laboratories, Inc. Alle rechten voorbehouden.