


IDEXX Web PACS 3.18 introduces:


# Enhanced dental features and integration

Now with full-mouth view and custom options

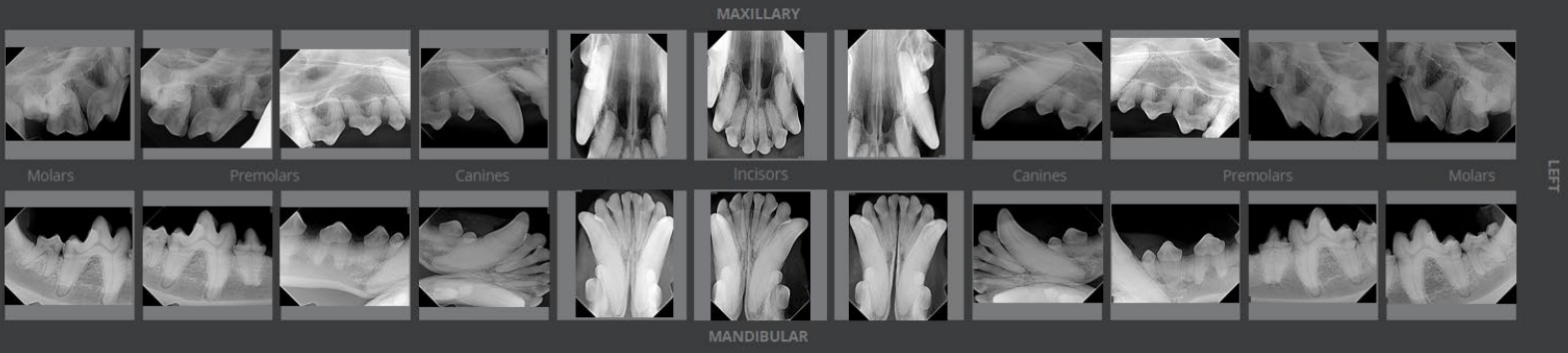

**Dental thumbnails open automatically in a full-mouth, dental-chart view.**

Add, replace, and move images. Switch to a row view with a single click.

**Click full-screen**  to see the largest possible view of a single image with no menus or browser tabs. Press **Escape** or click the arrows again to return to standard mode.

**Display the next full-screen image with a single arrow-click** .

**Customize image thumbnails.** Select thumbnail size, hide/show labels, and adjust the size of the thumbnail pane.

**Use the Reference Image Library** to compare your images with normal dental views.

IDEXX Web PACS\* can connect to your dental imaging system through most leading PIMS. Order images, view radiographs, capture charges, and save to the patient record seamlessly.

To learn more, click , select **Help**, then search for **Dental**.

**Note:** These options are not available at the basic subscription level. Contact an IDEXX consultant at 1-855-627-6573 for more information.

**IDEXX**

© 2024 IDEXX Laboratories, Inc. All rights reserved. • 2691135-00

\*IDEXX Web PACS is a trademark or registered trademark of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries. All other product and company names and logos are trademarks or registered trademarks of their respective holders. The IDEXX Privacy Policy is available at [idexx.com](https://www.idexx.com).