


Catalyst Dx Chemistry Analyzer


Quick Reference Guide

Note: For detailed instructions on using your Catalyst Dx™ Chemistry Analyzer, see the *IDEXX Catalyst Dx™ Chemistry Analyzer Operator's Guide*.

Catalyst Dx Analyzer Components

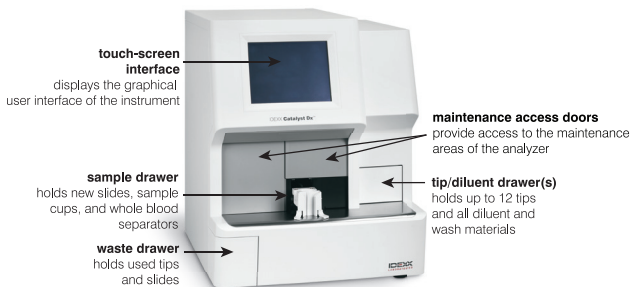

Sample Volume Required

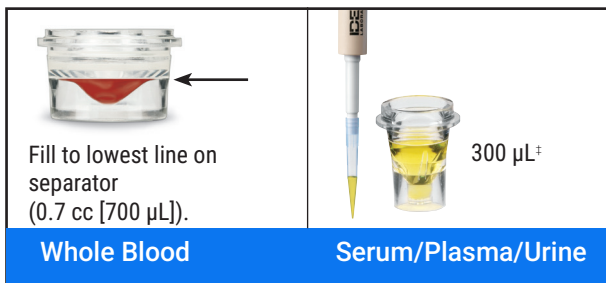

[‡] The sample volume required varies depending on the number of slides being run. For more information, see the Catalyst One Operator's Guide.

Running a Sample

1. Enter the sample information on the IDEXX VetLab™ Station.
2. Once the sample information appears in the Pending list on the Catalyst Dx Home screen, tap it and then tap **Select**.
3. Select the **Sample Type**.
4. If you are running a NH₃ slide, UPC slides, or PHBR slide, select the applicable check box.
5. Specify the desired type of dilution, and then use the up/down arrows to specify the desired dilution factor (total parts).
6. Tap **Next**.
7. Load the slides and sample in the sample drawer.
Note: If you are running a Lyte 4 CLIP, be sure to load it in the sample drawer before any other CLIPs or slides.
8. Tap **Run**. When the analysis is complete, the results are sent to the IDEXX VetLab Station automatically.

Dilutions

The Catalyst Dx analyzer supports automated dilutions (the analyzer mixes the sample and diluent for you) and manual dilutions (you prepare the dilution outside of the analyzer).

Note: Do not dilute samples that are undergoing NH₃, PHBR, FRU, TT₄, SDMA, PL, PROG, BA, CORT, or electrolyte testing. Do not perform an automated dilution on CRP samples.

Special Slides (NH₃, UPC, or PHBR)

- If you are running a UPC panel or a PHBR slide, do not load any other CLIPs or slides in the sample drawer.
- If you are running a NH₃ slide, be sure to load it in the sample drawer before any other CLIPs or slides.
- If you are running a UPC panel, be sure to load both the UCRE and UPRO slides into the sample drawer.
- If you are running a special slide without selecting the applicable special slides check box and/or you do not follow the on-screen instructions, your results will be flagged and you may receive inaccurate results.

Opening the Maintenance Access Doors

1. Press up on the door panel below the touch screen.
2. **If the area above the door panel has vertical plastic slats** (see photo 2a below), push down firmly on the door panel. The maintenance access doors are released.
OR
If the area above the door panel has a metal handle (see photo 2b below), pull down on both sides of the metal handle above the door panel until you hear a click. The maintenance access doors are released.
3. Place a finger beneath the center of the maintenance access doors and push up until the doors lock in place.



Closing the Maintenance Access Doors

1. **If the area above the door panel has vertical plastic slats** (see photo 1a below), push and hold down on the door panel. The maintenance access doors close automatically.
OR
If the area above the door panel has a metal handle (see photo 1b below), pull down on both sides of the metal handle above the door panel until you hear a click. The maintenance access doors close automatically.
2. Press up on the door panel below the touch screen until it clicks.



Cleaning and Maintenance

Monthly

- Clean optics windows, incubator ring, and white reference tile.
- If there is sample on the centrifuge shield, remove the shield from the analyzer and gently clean it with mild soap and water to remove the residue. Rinse, dry thoroughly, and replace.
- Run quality control on the analyzer after cleaning.

Quarterly


Remove the fan filter and vacuum it thoroughly to remove dirt or dust.

As Needed


- Upgrade your analyzer software promptly.
- Clean the internal components when prompted by the analyzer.
- Clean the outside of the analyzer with a damp (not wet) lint-free cloth. A mild liquid soap will remove grease.
- Always disconnect the power cable before cleaning the analyzer.
- If the screen gets dirty, apply an antistatic screen cleaning agent (NOT ammonia-based) to a clean cloth or paper towel and wipe the screen. Do not spray the cleaner directly onto the screen.
- Clean the interior of the waste drawer with a lint-free cloth dampened with 70% isopropyl alcohol.

To Clean the Internal Components


1. On the Catalyst Dx touch screen, tap **Tools** and then tap **Clean Analyzer**.
2. Open the maintenance access doors.
3. Remove the black whole blood separator carriers, any whole blood separators or sample cups from the sample drawer, and the white centrifuge shield. Then, clean the black carriers and white shield with an IDEXX-supported alcohol prep pad and return them to their positions.

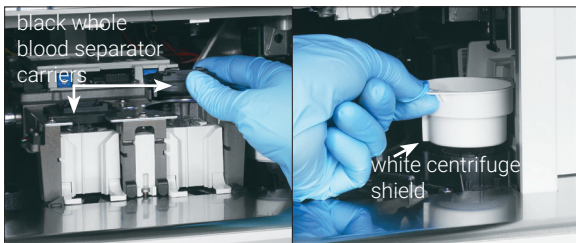

4. Lift the sample drawer assembly by pressing the center latch and lifting up.

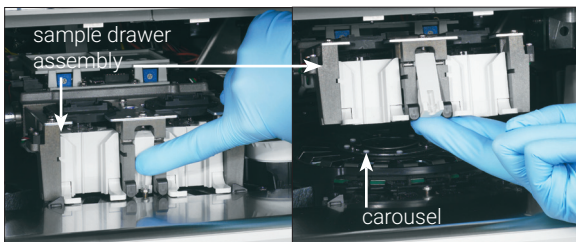

5. Remove the carousel by lifting up.
6. Using an IDEXX-supported alcohol prep pad, wipe the incubator ring track in a counterclockwise direction (do not wipe the optics or ion windows at this time). Repeat this step at least three times using a new alcohol prep pad for each wipe.
7. Clean the optics, ion windows, and reference tile on the carousel using the instructions in step 6.

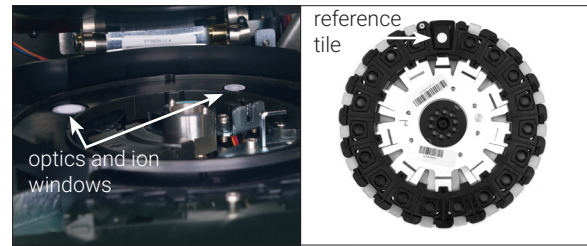

8. Using an IDEXX-supported dry optical tissue, dry the optics, ion windows, and reference tile, ensuring all signs of dampness have evaporated from the cleaned components. If streaks or smudges remain, repeat the cleaning process.
9. Replace the carousel.
10. Lower the sample drawer assembly and ensure that it is locked into place.
11. Close the maintenance access doors.
12. On the Catalyst Dx touch screen, tap **Done**. The analyzer initializes (approximately 5 minutes).

Running Quality Control

1. Tap the **Catalyst Dx** icon on the IDEXX VetLab Station Home screen.
2. Tap **SmartQC**.
3. Tap **Run SmartQC**.
4. Follow the on-screen instructions on the Catalyst Dx touch screen for loading the Catalyst™ SmartQC™ Control materials and completing the run.

IMPORTANT: Catalyst SmartQC should not be run with any samples or other CLIPs/slides.

Catalyst CLIPs and Slides



Chemistry	Abbreviation	Chem 18 CLIP	Chem 17 CLIP	Chem 16 CLIP	Chem 15 CLIP	Chem 11 CLIP	Chem 10 CLIP	Equine 15 CLIP	NSAID 7 CLIP	NSAID 6 CLIP	UPC Panel†	Lyte 4 CLIP	Individual Slides
Albumin	ALB	✓	✓	✓	✓	✓	✓	✓					✓
Alkaline phosphatase	ALKP	✓	✓	✓	✓	✓	✓	✓	✓	✓			✓
Alanine aminotransferase	ALT	✓	✓	✓	✓	✓	✓		✓	✓			✓
Amylase	AMYL	✓	✓										✓
Aspartate aminotransferase	AST							✓	✓	✓			✓
Bile acids	BA												✓
Blood urea nitrogen	BUN	✓	✓	✓	✓	✓	✓	✓	✓	✓			✓
Calcium	Ca	✓	✓	✓	✓			✓					✓
Cholesterol	CHOL	✓	✓	✓	✓								✓
Cortisol‡	CORT												✓
Creatine kinase	CK							✓					✓
Creatinine	CREA	✓	✓	✓	✓	✓	✓	✓	✓	✓			✓
Chloride	Cl											✓	
C-reactive protein‡	CRP												✓
Fructosamine‡	FRU												✓
Gamma-glutamyltransferase	GGT	✓	✓	✓	✓			✓					✓
Glucose	GLU	✓	✓	✓	✓	✓	✓	✓					✓
Potassium	K											✓	
Lactate	LAC												✓
Lactate dehydrogenase	LDH							✓					✓
Lipase	LIPA	✓	✓										✓
Magnesium	Mg												✓
Sodium	Na											✓	
Ammonia	NH ₃												✓
Phenobarbital‡	PHBR												✓
Inorganic phosphate	PHOS	✓	✓	✓	✓								✓
Pancreatic lipase‡	PL												✓
Progesterone	PROG												✓
Symmetric dimethylarginine‡	SDMA	✓		✓		✓			✓				✓
Total bilirubin	TBIL	✓	✓	✓	✓			✓					✓
Total protein	TP	✓	✓	✓	✓	✓	✓	✓					✓
Total T ₄ †	TT4												✓
Triglycerides	TRIG												✓
Urine creatinine	UCRE										✓		
Urine protein	UPRO										✓		
Uric acid	URIC												✓

†Validated reference intervals for equine and “other” species are unavailable.

‡Validated reference intervals for feline, equine, and “other” species are unavailable.