

4 tips for running the Catalyst Cortisol Test on your Catalyst Dx Chemistry Analyzer

The Catalyst™ Cortisol Test has a special workflow that saves you time by automatically:

- + Providing comprehensive interpretive comments to help determine the best next steps.
- + Grouping multistep protocol results together in the patient's record so they can be reviewed at a glance.

Remember these important tips to ensure you're accessing these time-saving benefits:

1

Select the Cortisol option on the Catalyst Dx™ analyzer touch screen, when prompted.

This initiates the special Catalyst Cortisol workflow.

2

Select the desired protocol and its additional protocol steps at the end of the run when prompted.*

3

Initiate runs for remaining tests in a multistep protocol using the [Add Test > Append](#) workflow.

This ensures multistep protocol results are grouped and provides comprehensive interpretive comments.

4

Review the results and interpretive comments to determine the best next steps.

Multistep protocol results that followed step 2 above will be grouped together with the most recent result and the comprehensive interpretive comments at the top.

*Forget to add a protocol/protocol step? Tap the patient in the In Process list, tap **Add Details**, and then select the patient's protocol information.