


4 tips for running the Catalyst™ Cortisol Test on your Catalyst One™ Chemistry Analyzer

The Catalyst™ Cortisol Test has a special workflow that saves you time by automatically:

- + Providing comprehensive interpretive comments to help determine the best next steps.
- + Grouping multistep protocol results together in the patient's record so they can be reviewed at a glance.
- + Queuing each test in a multistep protocol for easier test initiation.

Remember these important workflow tips to ensure you're accessing these time-saving benefits:

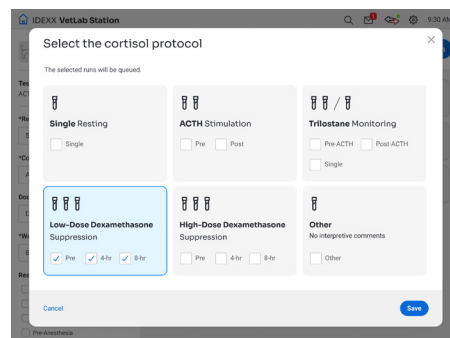

1

On the IDEXX VetLab™ Station Select Instruments screen, select Catalyst One and then select the Cortisol option.

This initiates the special Catalyst Cortisol workflow.

2

Select the desired protocol and its additional protocol steps when prompted (see the screen above).

Doing so automatically adds any additional runs for multistep protocols (e.g., ACTH Stimulation) to the In Process list for easier run initiation.

3

Initiate runs for remaining tests in a multistep protocol from the In Process list on the IDEXX VetLab Station Home screen (tap the next available test, tap Queued, and then tap Start Run).

This ensures multistep protocol results are grouped and provides comprehensive interpretive comments.

4

Review the results and interpretive comments to determine the best next steps.

Multistep protocol results that followed step 2 above will be grouped together with the most recent result and the comprehensive interpretive comments at the top.