

What's New in IDEXX-PACS 4.2 Imaging Software

Updates and enhancements

New, improved database: The IDEXX-PACS* 4.2 Imaging Software includes newer database technology for improved performance.

IMPORTANT: When you upgrade your software, click **Install** if you see a prompt to install Microsoft* Visual C++* Redistributable Packages. Microsoft Visual C++ is needed for the MYSQL drivers used in the new database.

Specifications and Training

Statement of intended use

The IDEXX Diagnostic Imaging Systems are intended for veterinary use only; they are not intended for human diagnostic use. For a detailed list of species and settings, contact IDEXX Diagnostic Imaging Customer Support at 1-877-433-9948.

Operating system requirements

Operating system requirements are slightly different for computers used to capture images with IDEXX Diagnostic Imaging systems and computers used for viewing images and using other software features:

- When capturing images, use only computers that are supplied by IDEXX.
- When viewing images and using other features (except image capture), use a computer running the Windows* 7 Professional, Windows* 8, Windows* 10, or Windows* 2012 Server operating system.

The Patient CD can be used with home or business editions of Windows operating systems. The Patient CD is not compatible with the Mac* OS X* operating system.

Minimum hardware requirements

The computer used to run the IDEXX-PACS software must meet these minimum requirements:

- 2 GHz dual-core 32-bit processor
- 2.0 GB RAM
- 20 GB free space on hard drive
- CD/DVD burner
- 128 MB graphics card
- 100 MB Ethernet port

To capture images with IDEXX Diagnostic Imaging systems, use only computers provided by IDEXX.

Learning to use your IDEXX-PACS software

We've created interactive tutorials and short training snippets that are the next best thing to having an IDEXX representative by your side. Visit the [Diagnostic Imaging Education and Training](#) page or go to idexxlearningcenter.com and click Snippets or Tutorials.

What's New in IDEXX-PACS* 4.2 Imaging Software



Proprietary rights notice

Information in this document is subject to change without notice. Companies, names and data used in examples are fictitious unless otherwise noted. No part of this document may be reproduced or transmitted in any form or by any means, electronic, mechanical or otherwise, for any purpose, without the express written permission of IDEXX Laboratories. IDEXX may have patents or pending patent applications, trademarks, copyrights or other intellectual or industrial property rights covering this document or subject matter in this document. The furnishing of this document does not give a license to these property rights except as expressly provided in any written license agreement from IDEXX Laboratories or an affiliate.

© 2016 IDEXX Laboratories, Inc. All rights reserved. • 06-0003399-05

*IDEXX-PACS is a trademark or registered trademark of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries. Microsoft, Windows, Windows Server, and Visual C++ are trademarks of Microsoft Corporation, registered in the United States and other countries. Mac and OS X are trademarks of Apple, Inc. registered in the United States and other countries. All other product and company names and logos are trademarks of their respective holders.