




CardioPet ECG Device

Quick Reference Guide





For software version 6.3 or earlier



Record and submit a CardioPet* ECG

1. Click **Select & Manage Patient** , and then choose or add the patient.
2. Attach CardioPet* comfort clips to the patient.
3. Turn on the CardioPet* ECG device.
4. Click **Start ECG**  to begin the live ECG stream. This activates wireless communication; the light on the device will illuminate.
5. Click **Record ECG** ; a timer appears in the upper left corner of the window.

Note: Recording stops automatically after 30 or 60 seconds (configurable in **Settings**).

Important: Clicking **Record** again will interrupt the recording.
6. Click **Stop ECG**  to stop the live ECG stream.
7. Click any of the following to send the ECG for interpretation:
 - **Upload and Finish Now** .
 - **Upload and Finish Later** .
 - **Send to PACS** , and then from the IDEXX-PACS* Imaging Software send the ECG to VetMedStat*.
8. Complete the case submission on vetmedstat.com.

For additional help, visit idexx.com/telemedicineresources or call 1-800-726-1212.