

IDEXX-PACS Keyboard and Mouse Shortcuts

Keyboard and mouse shortcuts can make it easier to interact with your capture PC, saving you time and effort as you work with your IDEXX Digital Imaging System using IDEXX-PACS Imaging Software.

Shortcuts are available on the **Image Capture** and **Image Viewer** windows.

| Keyboard | Mouse | Result |
|-----------------|---------------------------------|---|
| Press A | | Rotate image 90° clockwise |
| Press S | | Rotate image 90° counterclockwise |
| | Left-click + move left to right | Change the window (brightness) |
| | Left-click + move up and down | Change the level (contrast) |
| | Double-left-click | Restore original window and level |
| | Right-click + move up and down | Zoom |
| | Double-right-click | Fit image to window |
| Hold down Shift | Right-click | Magnify an area of the image. Continue holding Shift and the right mouse button to move the magnified area. |
| Hold down M | Right-click + move | Move around an image, if you have zoomed so that the image is larger than the screen. (Just like the Pan tool.) |

To Turn Shortcuts Off or On:

1. Select **Tools > Settings > System** on the menu bar at the top of the screen, then select **User Settings** on the left. In Cornerstone* Practice Management Software, select **Controls > Defaults > Imaging** on the menu bar at the top of the screen, then select **Keywords** on the left.
2. Check **Enable Imaging Shortcuts** to turn the shortcuts on. Uncheck it to turn the shortcuts off.
3. Click **Save**.