



# IDEXX-PACS\* 4.2 Imaging Software

The Newest Version of IDEXX-PACS\*

# IDEXX-PACS 4.2 Imaging Software

## Home window

Same familiar work flow, but with a new look—and it's easier to use.



IDEXX-PACS\* 3.7



IDEXX-PACS 4.2

# IDEXX-PACS 4.2 Imaging Software

## Home window

Search for images multiple ways.

Select date range or search.

Filter results 3 ways.

Select a pending request.

The screenshot shows the IDEXX Capture 4.2 software interface. At the top, there are tabs for 'Last 2 Days', 'This Week', 'Last Week', 'Date Range', and 'Reset'. A red arrow points to the 'Date Range' tab with the text 'Select date range or search.' Below these tabs, there are three dropdown menus: 'All Species', 'All Doctors', and 'All Modalities'. A red arrow points to these filters with the text 'Filter results 3 ways.' To the right of these filters, there is a search bar and a 'Capture' button. A red arrow points to the search bar with the text 'Select date range or search.' Below the search bar, there is a 'Pending' button. A red arrow points to this button with the text 'Select a pending request.' The main area of the interface shows a list of results, including 'Max (11/10/16)', 'Callin (11/10/16)', 'Maggie Mae (10/10/16)', and 'Stubbers (11/10/16)'.

# IDEXX-PACS 4.2 Imaging Software

## Home window

All your most recent cases are available on the Home window for quick reference.



To view details, click the bordered area.

# IDEXX-PACS 4.2 Imaging Software

Patient information

 Tabitha (15379)

Doe, Jess (9117)

American Shorthair,

Female, 3 Years





2/24/17  
3:02 PM



Displays study

Capture a new image.

< Return



# IDEXX-PACS 4.2 Imaging Software

## Visual shot tree

Choose shots using the new species-specific visual shot tree.




IDEXX-PACS 3.7



IDEXX-PACS 4.2

# IDEXX-PACS 4.2 Imaging Software

Select from your favorite shots.

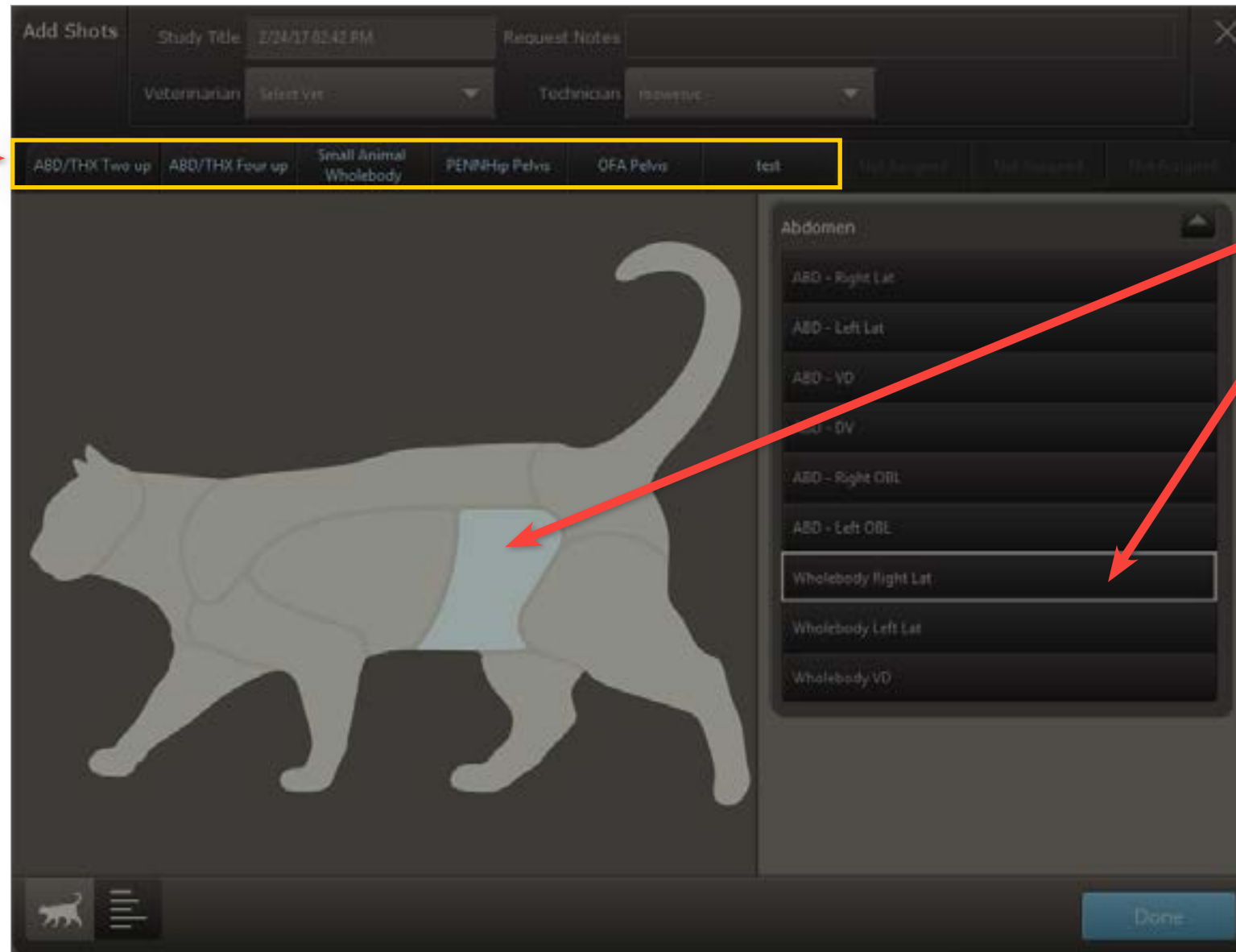


Or select a region, then shots.


# IDEXX-PACS 4.2 Imaging Software

## Capture window

Similar work flow, but with new tools to make image capture easier.

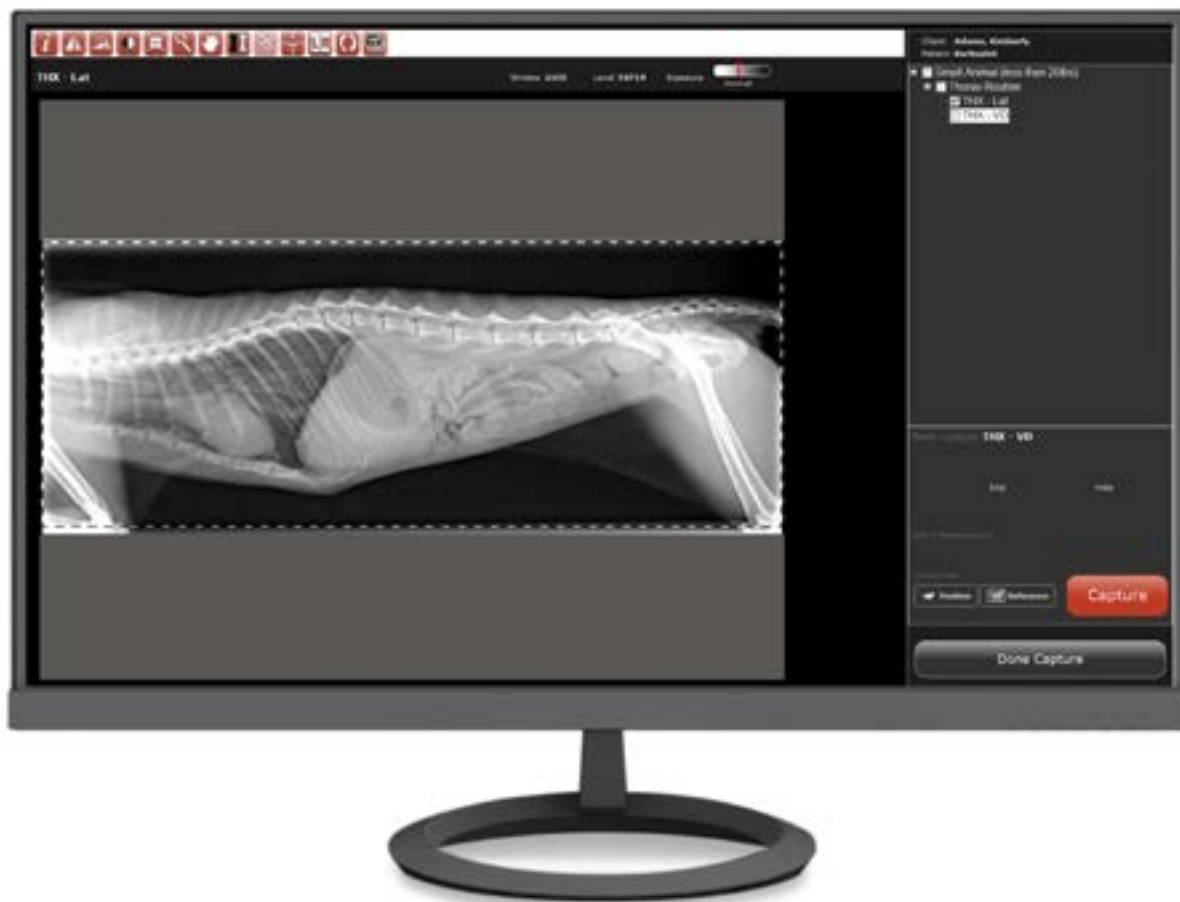

IDEXX-PACS 3.7

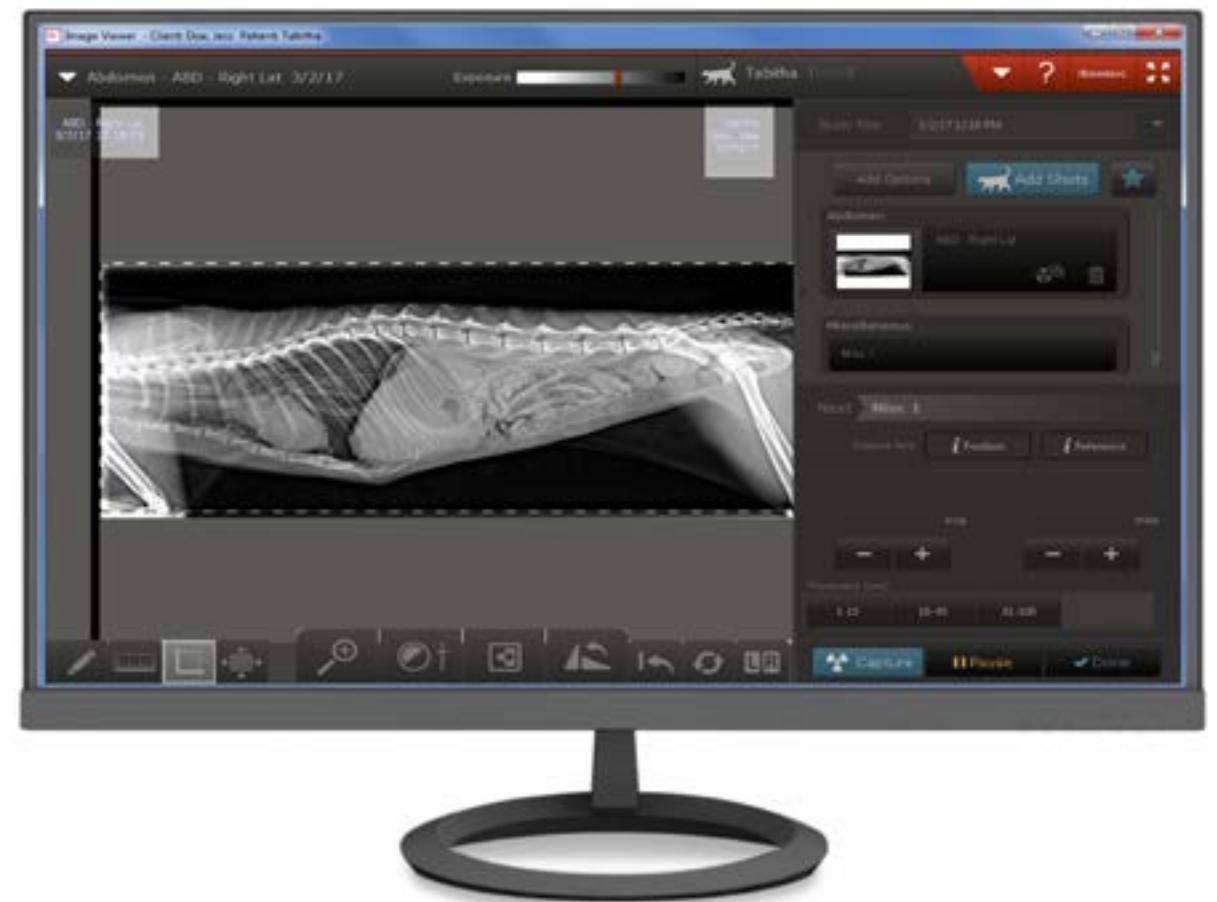


IDEXX-PACS 4.2



# IDEXX-PACS 4.2 Imaging Software

Add specialized shot options.

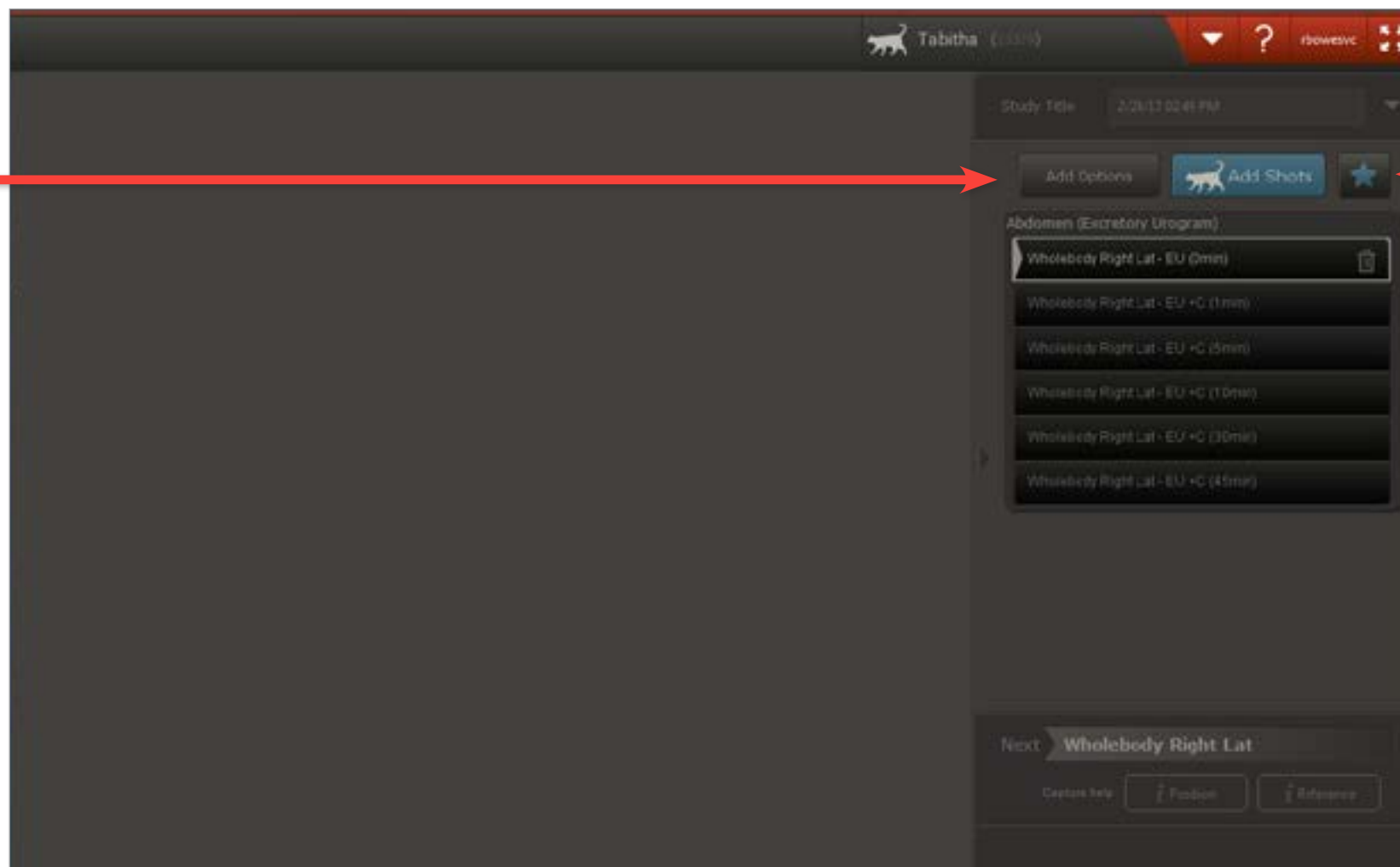


Save shots as a favorite.

To view details, click the bordered areas.

# IDEXX-PACS 4.2 Imaging Software


Choose options  
for the selected  
anatomy.

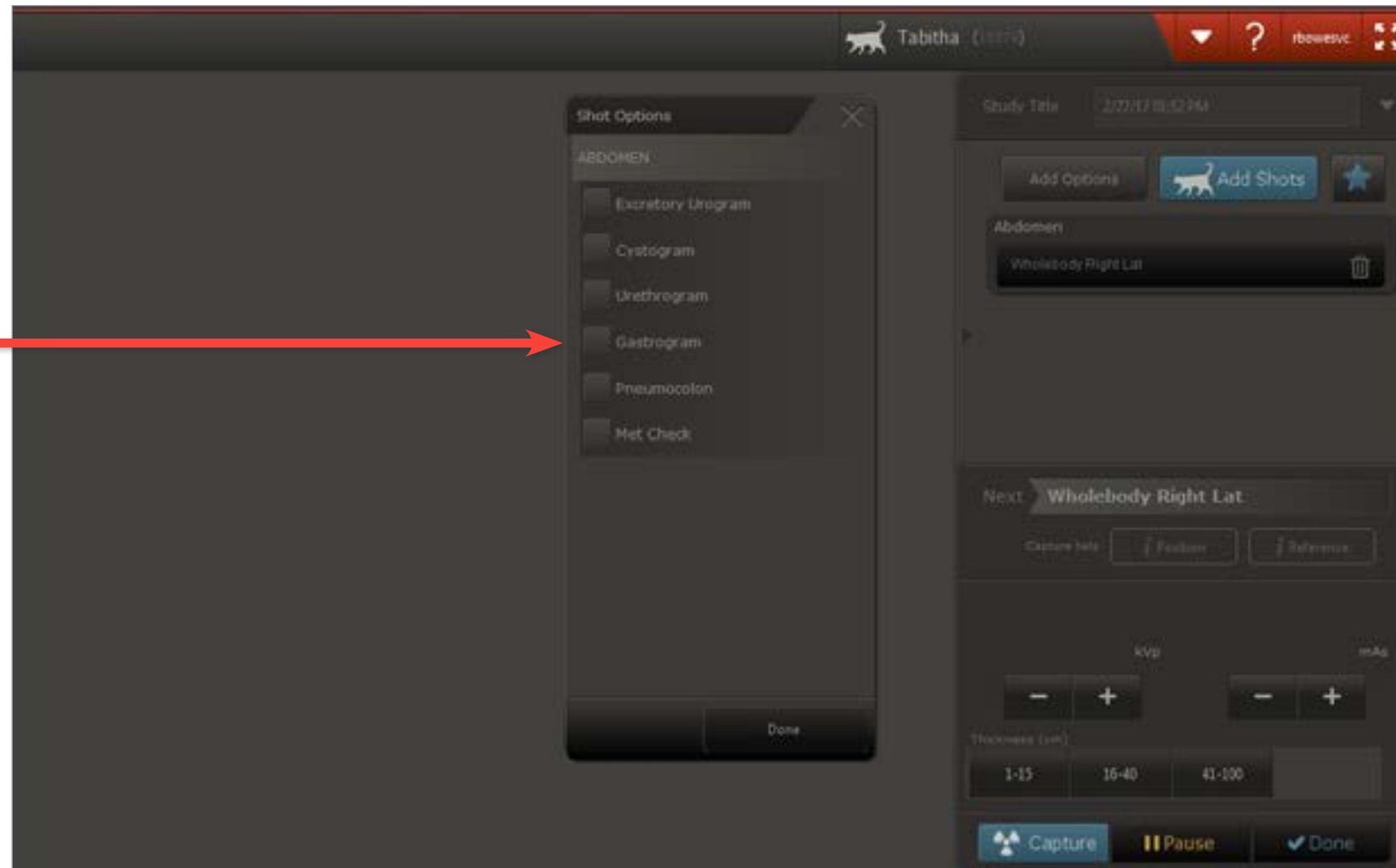

< Return

# IDEXX-PACS 4.2 Imaging Software

Save current shot list as a favorite.

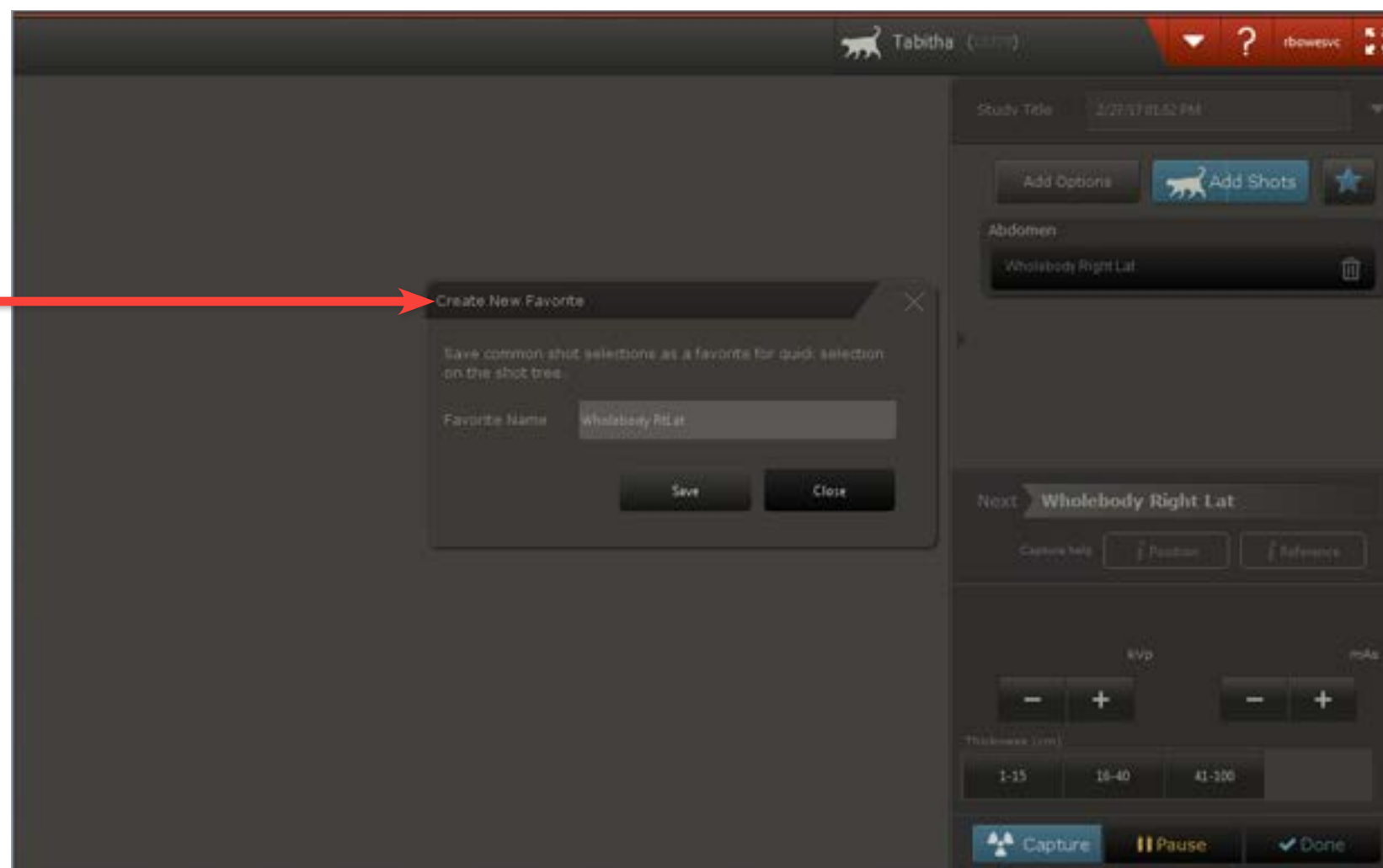

< Return

# IDEXX-PACS 4.2 Imaging Software

## Image viewer

View and enhance images using the tools you've come to rely on.




IDEXX-PACS 3.7



IDEXX-PACS 4.2

# IDEXX-PACS 4.2 Imaging Software

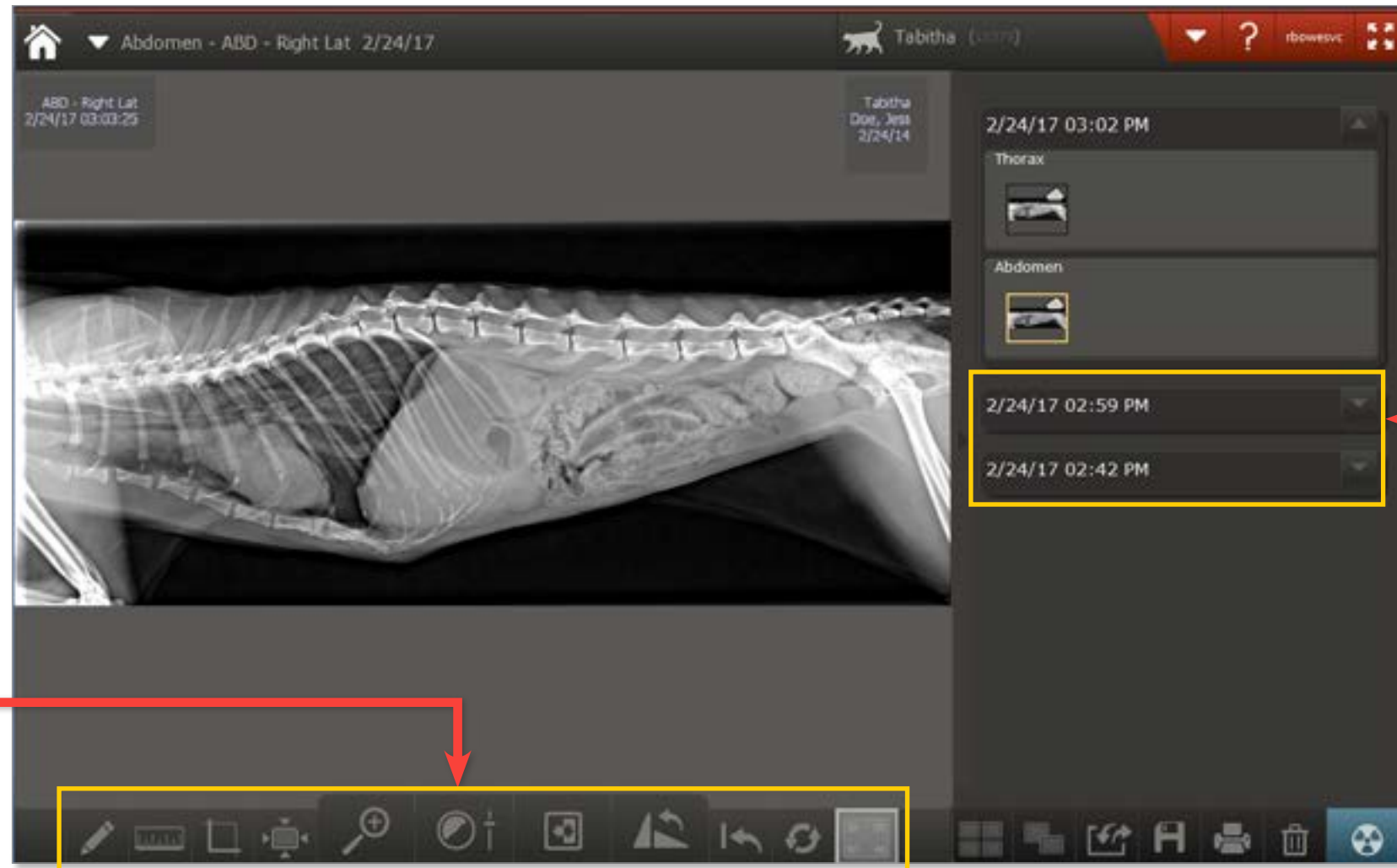


Familiar tools to  
enlarge, zoom,  
adjust, and measure

Previous  
studies are  
available  
for quick  
reference.

# IDEXX-PACS 4.2 Imaging Software

Redisplays  
Home window

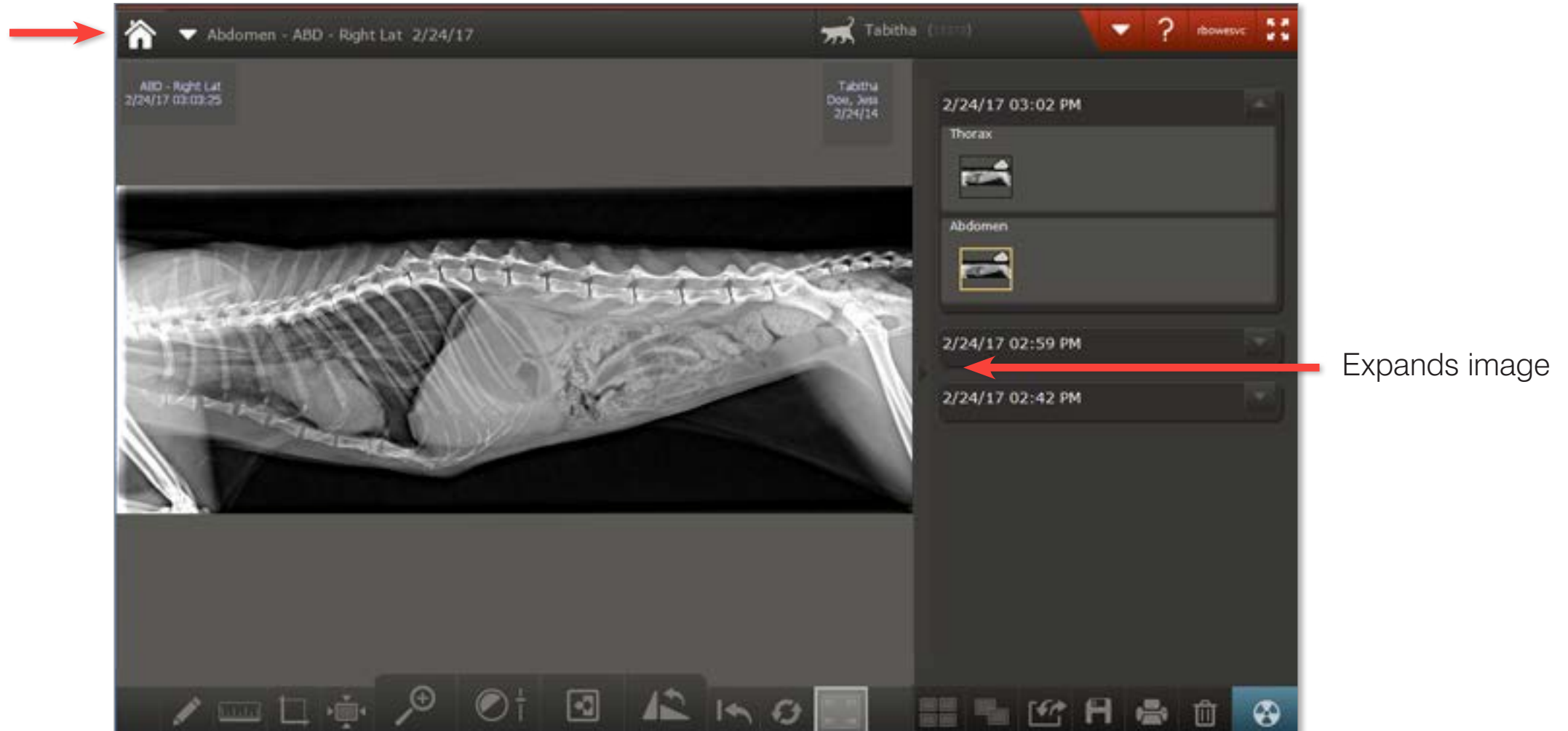


To see interaction, click the bordered areas.



# IDEXX-PACS 4.2 Imaging Software

---

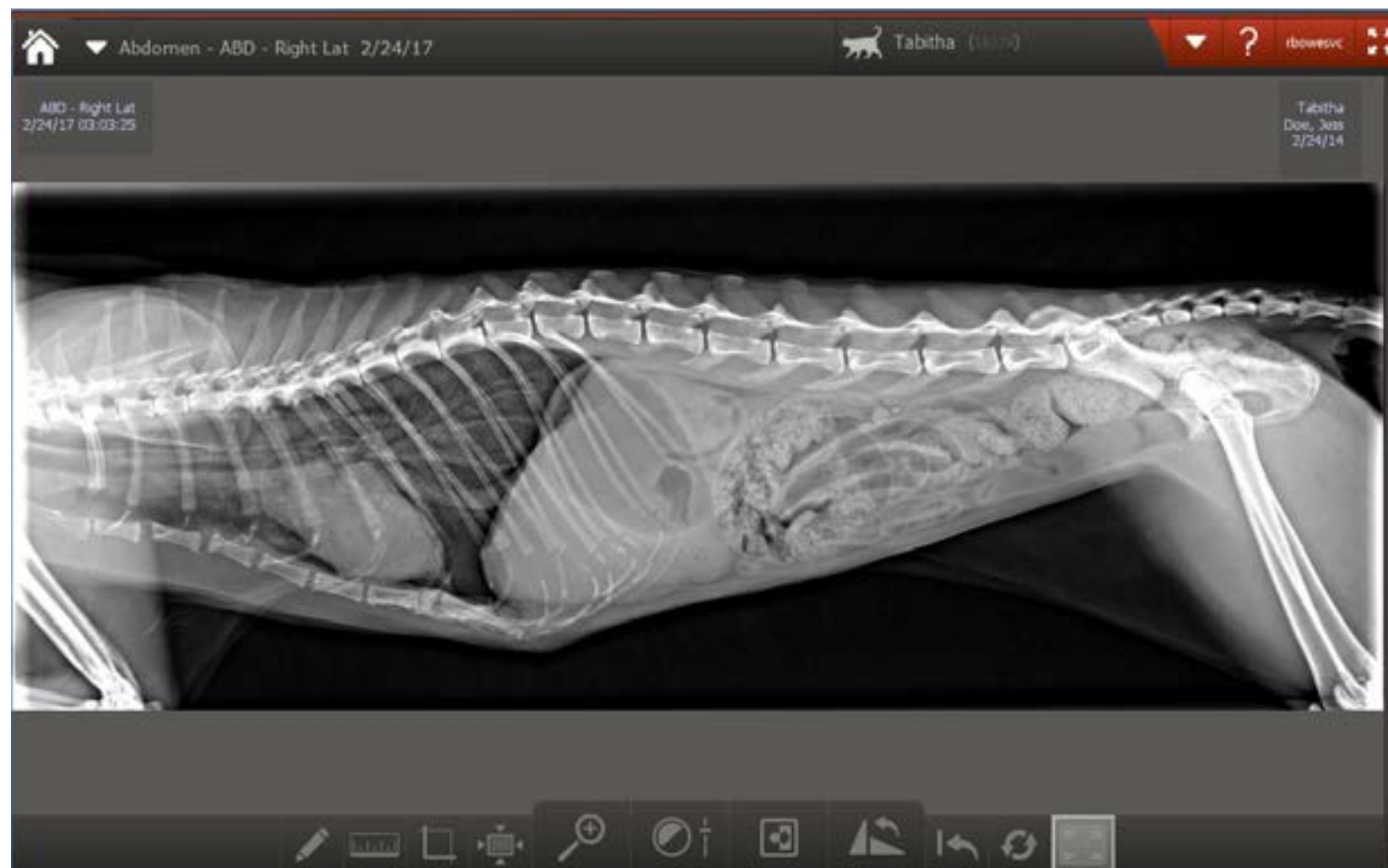

< Return

# IDEXX-PACS 4.2 Imaging Software



< Return

# IDEXX-PACS 4.2 Imaging Software

## IDEXX Web PACS compatible


IDEXX-PACS 4.2 is compatible with IDEXX Web PACS, our next-generation cloud PACS solution.

With an IDEXX Web PACS subscription, your images are sent to the cloud automatically and become available on any internet device.

You'll find familiar tools and even more advanced features:

- Instant software upgrades
- Advanced measurement tools
- Unmatched collaboration and communication capabilities


To learn more or see a demonstration, contact an IDEXX Digital Radiography Specialist.

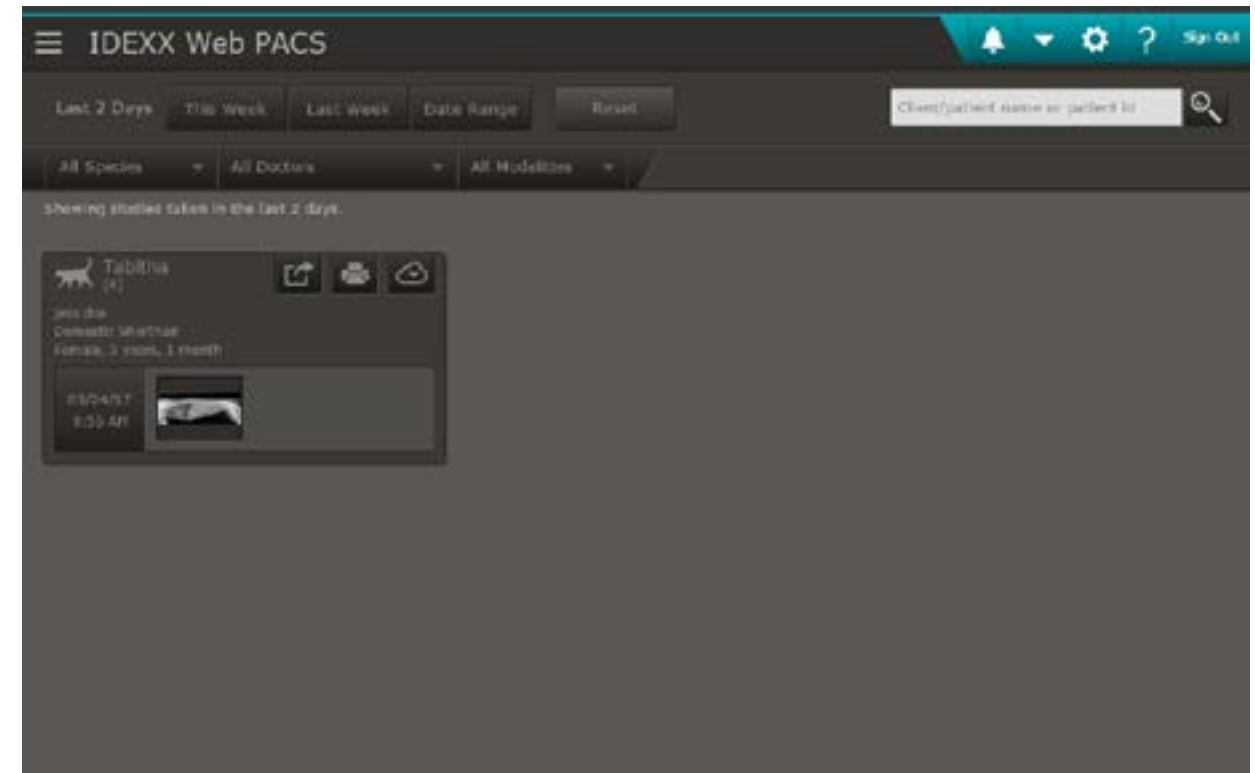

IDEXX Web PACS Home page

**To see interaction, click the bordered areas.**


# IDEXX-PACS 4.2 Imaging Software

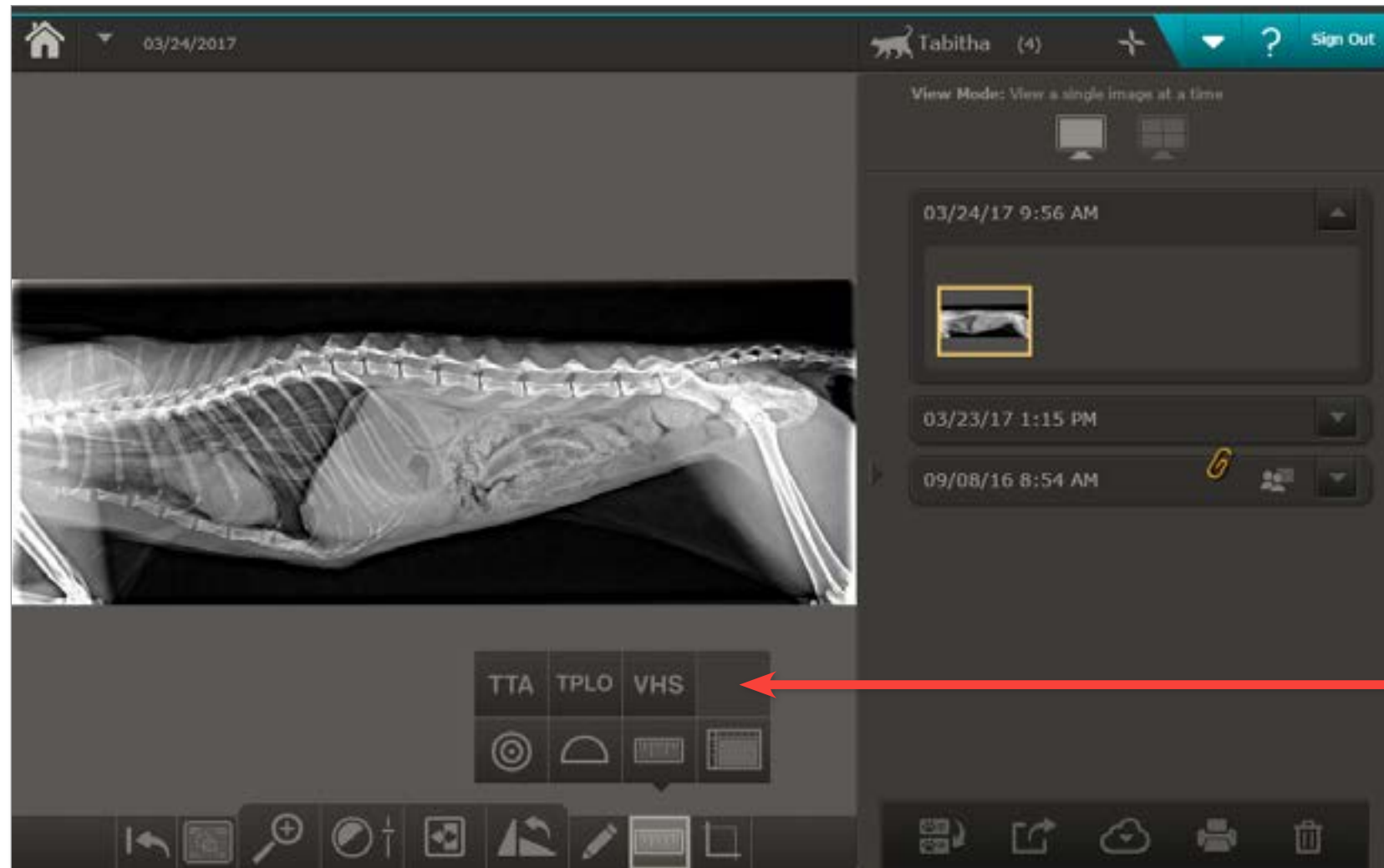

Advanced tools

[Return](#)

# IDEXX-PACS 4.2 Imaging Software

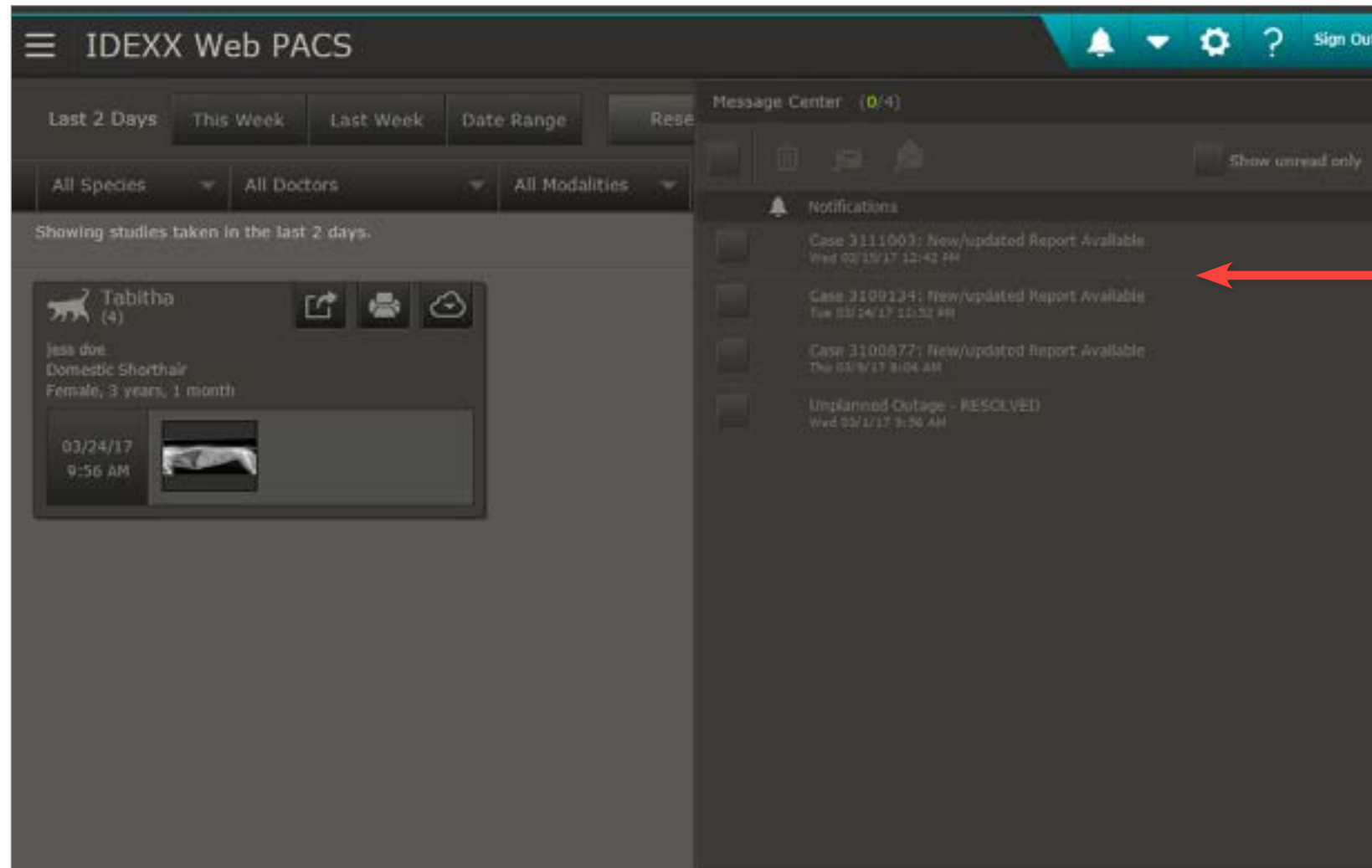

Message center alerts you to new telemedicine reports, software updates and more.

< Return

For more information, visit your IDEXX-PACS Resources  
page at [idexx.com/PACSresources](https://www.idexx.com/PACSresources)

