

SNAP FIV/FeLV Combo Test

Test accuracy

Sensitivity and specificity of the SNAP™ FIV/FeLV Combo Test

Analyte	Reference method	SNAP test result		Total	Sensitivity (95% CL)
		+	-		Specificity (95% CL)
FeLV ¹	+	158	0	407	100% (97.7%–100%)
	-	2	247		99.2% (97.1%–99.9%)
FIV ²	+	95	0	279	100% (96.2%–100%)
	-	4	180		97.8% (95.4%–99.4%)

Reference method

1. PetChek™ Feline Leukemia Virus Antigen Test
2. Feline immunodeficiency virus (FIV) Western blot

For interpretation and diagnostic algorithms, visit idexx.com/FelineComboResources.

Source: Singh S, Davenport KA, Schooley E, et al. Diagnostic accuracy of a point-of-care immunoassay for feline immunodeficiency virus antibodies, feline leukemia virus antigen, and *Dirofilaria immitis* antigen. *Viruses*. 2023;15(10):2117. doi:10.3390/v15102117

© 2026 IDEXX Laboratories, Inc. All rights reserved. • 09-2692745-00
All trademarks are owned by IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.
The IDEXX Privacy Policy is available at idexx.com.

