SNAP* Total T₄ Test Kit

For use only with the
IDEXX SNAPshot Dx* Analyzer.

For veterinary use only.

English Version

Overview


The SNAP Total T₄ Test Kit is an enzyme-linked immunosorbent assay (ELISA) for the quantitative measurement of total T₄ (thyroxine) in canine, feline (domestic) and equine serum or plasma.[†] For in vitro use only.

SNAP Total T₄ Test Ranges

Dynamic Range: 0.5–7.0 µg/dL (6.4–90.0 nmol/L)

Precautions and Warnings

- This test cannot be visually interpreted. The SNAPshot Dx* Analyzer must read all tests.
- Use a separate pipette tip and sample tube for each test.
- Do not use components after their expiration dates.
- Do not mix components from different lots of test kits.
- Be sure that two blue reagent spots are visible before adding the sample.
- Keep the device in its sealed foil pack at room temperature (18°C–27°C/64°F–80°F) for at least 15 minutes before use.
- Prewarming of the conjugate solution is not required.
- Keep the SNAP device horizontal during the entire test procedure.**
- IMPORTANT: Do not depress the activator until instructed.**
- Insert the device into the analyzer immediately after activation.

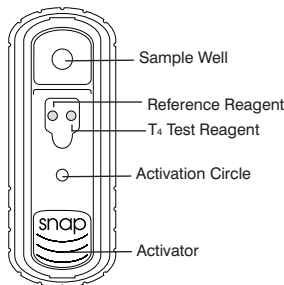Storage

- SNAP devices and test reagents must be stored at 2°C–8°C (36°F–46°F).
- Do not freeze, heat or microwave the kit components.
- Take out only the kit components you will need.
- Do not leave kit components at room temperature for more than 8 consecutive hours.

Kit Components

- Conjugate solution
- SNAP test devices
- Disposable pipette tips for the 300-µL manual conjugate pipette
- Disposable pipette tips for the 65-µL manual sample pipette
- Disposable sample tubes
- Workstation

Components Required (Not Provided in Kit)

The following required components are provided with the SNAPshot Dx Analyzer:

- 65-µL manual sample pipette
- 300-µL manual conjugate pipette

Sample Information

- Do not use whole blood. Use serum or plasma,[†] either fresh, previously frozen or stored at 2°C–8°C (36°F–46°F) for up to 7 days. To store longer than 7 days, freeze the sample (-20°C/-4°F or below). Remove serum or plasma[†] from the red blood cells before freezing. Bring previously refrigerated or frozen samples to room temperature (18°C–27°C/64°F–80°F) before running assays.
- SNAP Total T₄ sample size requirements: 65 µL.
- Centrifuge previously frozen or older samples before use.
- Results of this test are not affected by hemolysis or lipemia.

Test Procedure

The SNAPshot Dx Analyzer provides on-screen instructions throughout the analysis. For complete instructions, refer to the Using the SNAPshot Dx Analyzer chapter in the *IDEXX SNAPshot Dx Analyzer Operator's Guide*.

Species	Sample Volume	Conjugate Volume
Canine, Feline, Equine	65 µL	300 µL

IDEXX Technical Support

U.S./Canada/Latin America 1-800-248-2483
Europe 00800 1234 3399
Australia 1300 44 33 99
New Zealand 0800-102-084
Asia 886-2-2888-3336, 230

If possible, please have your kit lot number available.

[†]Plasma/lithium heparin

*SNAP and SNAPshot Dx are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

Patent information: idexx.com/patents.

© 2015 IDEXX Laboratories, Inc. All rights reserved • 06-13620-01

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© 2015 IDEXX Laboratories, Inc. Tutti i diritti sono riservati.