

Supplementary Guide to Assembling Balt Cart

Instructions to accommodate the IDEXX I-Vision CR® System

Recommended assembly

This document serves as a supplement to ensure your cart is ready for equipment installation. To assemble the cart, **please refer to the instructions enclosed within the box**. Since the positions of the shelves are totally adjustable, we recommend you use the following scheme while assembling the cart:

1. Secure the bottom shelf onto the cart sides by using 4 bolts, then secure the back to the bottom of the cart, according to the instructions.
2. Install wheels as outlined in the assembly instructions. (This will make installation of shelves easier because cart will be mobile.)
3. Attach keyboard bracket and support arms to the smaller of the shelves. This shelf may be placed on bracket with the lip facing either way (forward or backward). If the shelf is placed with the lip forward, it covers the keyboard brackets better; but if it's placed backward, it provides a channel through which wires can feed. Choose either direction for this shelf. Place the smaller shelf on the cart with the top holes in the bracket lined up no farther down than the third hole in the cart sides (arrows pictured in diagram at right are as far down as shelf can go).
4. Attach large shelf to the other two brackets: install these brackets with their top holes lined up 9 holes below the bottom of the shelf brackets you installed in step 3; this will leave 8 open holes between shelf brackets.
5. Do not install computer side tray. Tighten all screws and bolts, as necessary. Some small modifications to this setup can be used—just follow the key points, below, to ensure the cart will be ready to receive equipment upon installation.

Key points to remember

The scanner cart can be set up with a few minor variations. Just remember the following points:

- The top shelf (smaller of the two) can be placed on the bracket either way (forward or backward).
- The scanner needs to sit atop the large shelf to accommodate scanner depth (small shelf offers insufficient depth).
- The keyboard tray should go on the top shelf (smaller shelf); otherwise, the keyboard is inaccessible while plates are loaded into the scanner.
- The PC side tray should be avoided whenever possible. The PC fits well several places on the cart, and is less susceptible to animal fluids if not on the side tray.
- The scanner requires 14½ inches of height clearance, so be sure to allow 8 empty holes between shelf support arms.
- When in doubt, refer to the pictures and diagram on this page.



Above: Image of assembled cart

Below: Diagram of location of installed shelves

