


Easier, faster TPLO measuring

Enjoy greater efficiency when using the IDEXX Web PACS* TPLO tool. The tibial plateau leveling osteotomy (TPLO) tool now uses artificial intelligence to detect anatomical landmarks, place the TPLO lines where they belong, and display the tibial plateau angle. All in less than one second.

To calculate bone rotation, simply apply the saw blade indicator and select a blade size.

Tip: Find the TPLO tool under **Advanced Measurements**  > **TPLO**.

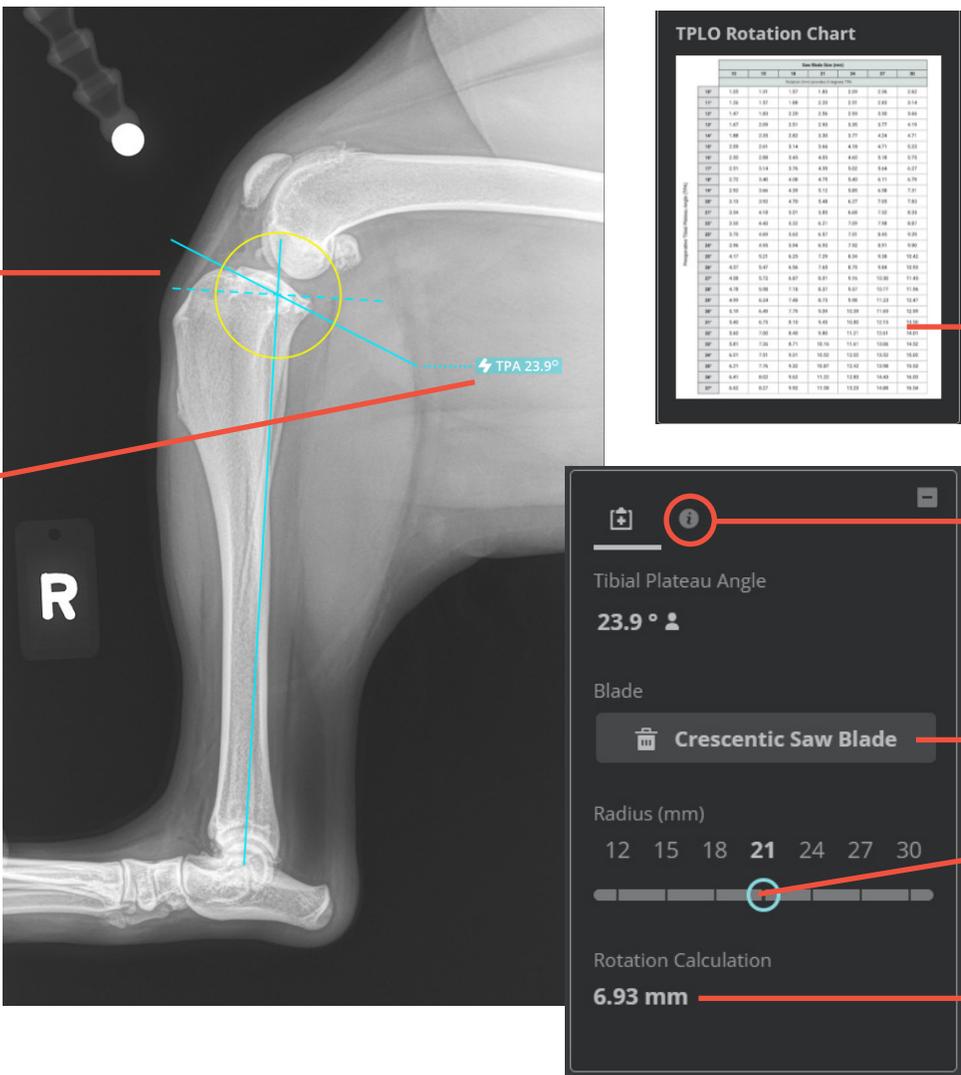

Detects anatomy and places lines automatically. Lines can be adjusted if needed.

Calculates the tibial plateau angle.

TPLO Rotation Chart

Tibial Plateau Angle (°)	Saw Blade Size (mm)									
	12	15	18	21	24	27	30	33	36	39
12°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
13°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
14°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
15°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
16°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
17°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
18°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
19°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
20°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
21°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
22°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
23°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
24°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
25°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
26°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
27°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
28°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
29°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
30°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
31°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
32°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
33°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
34°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
35°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
36°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
37°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
38°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
39°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00
40°	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00	0.00

Rotation chart

Links to the rotation chart and a positioning guide.

Click to apply a saw blade indicator and then select a radius.

Rotation value

Look for new features and enhancements in every IDEXX Web PACS update.