

# Enhanced dental features and integration

With full-mouth view and custom options



**Dental thumbnails open automatically in a full-mouth, dental-chart view.**

Add, replace, and move images. Switch to a row view with a single click.

**Click full-screen**  to see the largest possible view of a single image with no menus or browser tabs.

To return to standard mode, click  in the upper-right corner of the full-screen window or press **F11/Escape** on your keyboard.

**Display the next full-screen image with a single arrow-click** .

**Customize image thumbnails.** Select thumbnail size, hide/show labels, and adjust the size of the thumbnail pane.

**Use the Reference Image Library** to compare your images with normal dental views.

**IDEXX Web PACS can integrate most leading PIMS and dental imaging systems.**

Order images, view radiographs, capture charges, and save to the patient record seamlessly.

To learn more, click , select **Help**, and then search for **Dental**.

**Note:** These options are not available at the basic subscription level. Contact an IDEXX consultant at 1-855-627-6573 for more information.