

IDEXX-PACS Imaging Software

Release Notes

Updates and enhancements

- This version includes support for the new ImageVue DR50 Plus Diagnostic Imaging System, which includes:
 - Thickness as a required field for image capture.
 - Enhanced image processing.
 - Updated positioning image library.
-

Specifications and training

Statement of intended use

The IDEXX Diagnostic Imaging systems are intended for veterinary use only; they are not intended for human diagnostic use. For a detailed list of species and settings, contact IDEXX Diagnostic Imaging Customer Support at **1-877-433-9948**.

Operating system recommendations

Operating system recommendations and requirements are slightly different for computers used to capture images with IDEXX Diagnostic Imaging systems and computers used for reviewing images and using other software features:

- When capturing images, use **only** computers that are supplied by IDEXX.
- When reviewing images and using other features (except image capture), use a computer running the Windows 10 or higher Professional, Enterprise, or IoT Enterprise LTSC operating system.
- Make sure that all Windows updates have been installed on your IDEXX-PACS computers.

The Patient CD can be used with home or business editions of Windows operating systems. The Patient CD is not compatible with the Mac OS X operating system.

IDEXX-PACS Imaging Software

Release Notes

Minimum hardware recommendations

The computer used to run the IDEXX-PACS software should meet these minimum requirements:

RAM: 8.0 GB (Windows 64-bit operating system)

Intel dual-core processor or better

20 GB free space on hard drive

CD/DVD burner

100/1000 Mbps Ethernet port

Internet speeds:

– Upload: 5 Mbps

– Download: 10 Mbps

Monitor resolution:

– Minimum: 1024 x 768

– Recommended: 1280 x 1024 or higher

Contact IDEXX Diagnostic Imaging Customer Support if you need help upgrading RAM or freeing disk space.

To capture images with IDEXX Diagnostic Imaging systems, use only computers provided by IDEXX.

Learning to use your IDEXX-PACS software

We've created interactive tutorials and short training videos that are the next best thing to having an IDEXX representative by your side. Visit the [Diagnostic Imaging Education and Training](https://www.idexx.com/learning-center) page or go to idexxlearningcenter.com.



IDEXX Laboratories, Inc.
One IDEXX Drive
Westbrook, ME 04092 USA
1 207 556 0300
[idexx.com](https://www.idexx.com)

Proprietary rights notice

Information in this document is subject to change without notice. Companies, names, and data used in examples are fictitious unless otherwise noted. No part of this document may be reproduced or transmitted in any form or by any means, electronic, mechanical, or otherwise, for any purpose, without the express written permission of IDEXX Laboratories. IDEXX may have patents or pending patent applications, trademarks, copyrights, or other intellectual or industrial property rights covering this document or subject matter in this document. The furnishing of this document does not give a license to these property rights except as expressly provided in any written license agreement from IDEXX Laboratories or an affiliate.

© 2025 IDEXX Laboratories, Inc. All rights reserved. • 06-0003399-20

IDEXX-PACS and ImageVue are trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries. All other product and company names and logos are trademarks of their respective holders.

