Dear Valued Customer:

I'm pleased to send you the latest software upgrade for your SNAP Pro\* Analyzer—software version **2.8** is enclosed. This version includes:

- Improved stability in the connection between the SNAP Pro analyzer and the IDEXX VetLab\* Station.
- New language settings for German and Italian.

Software upgrades ensure that your IDEXX analyzers provide the consistent and accurate results you depend on. It is important that you upgrade at your practice's earliest convenience.

### To upgrade your analyzer:

**Note:** The upgrade process takes less than 6 minutes to complete.

1. Ensure the battery life indicator on the analyzer is showing 75% () or 100% () power and then plug it into an appropriate electrical outlet.
2. *IDEXX VetLab\* Station-connected users only:* disconnect your analyzer from the IDEXX VetLab Station by tapping **More** at the bottom of the Home screen, tapping **Connections**, tapping **No**, and then tapping the **Home** icon.
3. Tap **More** and then tap **About SNAP Pro**.
4. Tap **More** again and then tap **Update SNAP Pro**.
5. When prompted, insert the USB drive into one of the two USB ports on the back of the analyzer.
6. Wait 10 seconds and then tap **OK once**.  
**Note:** There is a slight delay after the button is tapped. Tapping the button multiple times can impact the upgrade process.
7. Tap **Yes** and then follow the on-screen instructions.
8. Remove the USB drive when prompted. The screen turns black, the analyzer reboots, and the screen and status light may turn on/off several times. The analyzer is ready for use when the Home screen appears.
9. *IDEXX VetLab Station-connected users only:* Reconnect the analyzer to the IDEXX VetLab Station once the SNAP Pro\* Home screen appears (tap **More** at the bottom of the Home screen, tap **Connections**, tap **Yes**, tap **Change Network**, select the **IDEXXw1** network, and then tap the **Home** icon).
10. Confirm the software was upgraded successfully by tapping **More** at the bottom of the Home screen, tapping **About SNAP Pro**, and then confirming the software version is 2.8.0. If there is a discrepancy, please contact IDEXX Technical Support.

Insert the USB drive into one of these ports

**Note:** Once the upgrade process is complete, the status light on the analyzer may remain off until a run is started or the instrument reboots.

We appreciate the opportunity to provide you with innovative products and services that help you deliver high-quality care and grow your practice. If you have any questions about this upgrade, please contact IDEXX Technical Support.

Regards,

Cameron Wise  
SNAP Pro\* Product Manager

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