
Alertys*

OnFarm Pregnancy Test

Test With Confidence™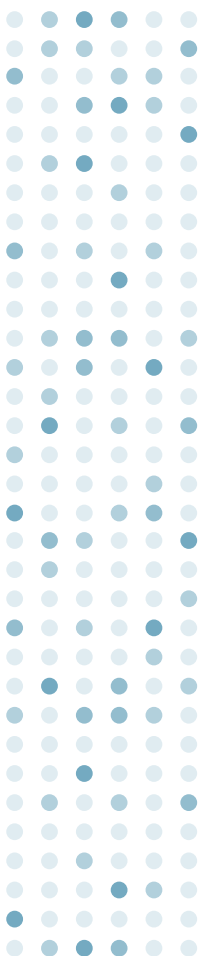

06-0013953-04

IDEXX

Alertys* OnFarm Pregnancy Test

Name and Intended Use

The IDEXX Alertys* OnFarm Pregnancy Test is a rapid method for the detection of pregnancy-associated glycoproteins (PAGs) in whole blood (EDTA), plasma (EDTA) or serum of cattle (cows and heifers), goats and sheep as a marker for pregnancy.

General Information

Accurate and timely detection of pregnancy in production animals is an essential component of today's reproductive management programs. Veterinarians and farmers use detection of non-pregnant (open) animals to make decisions regarding re-breeding and to manage calving intervals, thereby maximizing milk production and revenue for the farm. Confirmation during the stages of gestation and at dry-off are also very economically important. In addition, producers may check for pregnancy at the many stages of production.

Bovine Samples

Traditionally, pregnancy is determined through rectal palpation of the cow or by transrectal ultrasound (TU). The IDEXX Alertys* OnFarm Pregnancy Test provides users a method for accurate detection of pregnancy as early as 28 days post-breeding, and throughout gestation, which can give veterinarians, dairy farmers and beef producers an important tool for the early identification of pregnant cows. The test can be used ≥ 70 days post calving.

Goat and Sheep Samples

Accurate and timely detection of pregnancy in sheep and goat is an essential component of today's reproductive management programs. Veterinarians and farmers use early detection of non-pregnant (open) sheep and goat to enable faster rebreeding, thereby maximizing revenue for the farm. Traditionally, pregnancy is determined by abdominal ultrasound on sheep and goats. The Alertys* OnFarm Pregnancy Test provides users a method for accurate detection of pregnancy in goats from ≥ 28 days and in sheep ≥ 35 days post-breeding, and throughout gestation, which can give veterinarians and producers an important tool for early identification and management of open animals.

Precautions and Warnings

- Handle all biological materials as potentially infectious.
- Refer to the product Material Safety Data Sheet for additional information.
- Do not pipette by mouth.
- Do not eat, drink or smoke where sample or kit components are being handled.
- All wastes should be properly decontaminated prior to disposal. Dispose of contents in accordance with local, regional, and national regulations.
- Do not mix components from kits with different serial numbers.
- Open pouch immediately before testing. Do not use device if pouch is punctured or improperly sealed.
- Do not expose device to strong light before or during testing. Do not perform assay in direct sunlight.
- Do not use kit past expiration date.
- Do not reuse pipettes.
- Do not cross-contaminate samples.

Storage

- Store the Alertys* OnFarm pouched device at 2-30°C or 36-86°F. Devices are stable until expiration date, provided they have been stored properly.
- All components, including specimens, must come to 15-30°C or 59-86°F before running the test.
- Whole blood should be stored at 2-8°C or 36-46°F and cannot be stored more than 24 hours.
- Only open pouches for the number of devices to be run at that time. Once pouch has been opened, the device cannot be stored for later use.

Components

Test pouches , each include:
• 1 Alertys* OnFarm Pregnancy Device • 1 pipette • 1 single-use wash solution

Needed	Blood	Serum	Plasma
Sample collection tubes (or syringe) with EDTA anticoagulant - collection tube length should be no longer than 100mm	✓		✓
Sample collection tubes (or syringe) without anticoagulant		✓	
Blood collection needles	✓	✓	✓
Timer of any type (including phone)	✓	✓	✓
Centrifuge for processing serum/plasma		✓	✓


Sample Information

Reminder: All samples and devices must be at 15-30°C or 59-86°F before beginning the test procedure.


	Source	Storage
Whole Blood	Blood should be collected in tubes containing EDTA and mixed well by slowly inverting tube 10 times immediately after sample draw.	Short Term Only: Whole blood can be stored for no more than 24 hours at 2-8°C or 36-46°F. Blood cannot be allowed to freeze or partially freeze as this may compromise test results.
Serum	Blood should be collected in tubes without anti-coagulant and processed to serum by centrifugation. Alternatively, the tube may be allowed to sit upright until serum separates.	Short Term: Serum and plasma can be stored at 2-8°C or 36-46°F for up to 7 days. Long Term: Serum and plasma can be stored at or below -20°C or -4°F. All previously frozen serum and plasma specimens must be thoroughly mixed prior to removal of the volume to be assayed.
Plasma	Blood should be collected in tubes containing EDTA and mixed well by inversion immediately after sample draw. Tubes should be processed to plasma by centrifugation.	

Test Procedure (Blood, Serum and Plasma)


- 1 Place the Alertys* OnFarm Pregnancy test device on a clean flat surface. The device must be kept horizontal throughout the process to ensure accurate results. Write sample ID or stick ID label on the device surface. Keep single-use wash solution and pipette next to the device.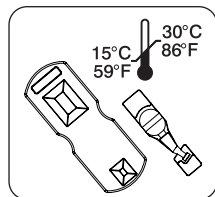


- 2 Mix whole blood or serum samples by inverting the tube. Fully squeeze the top bulb of the pipette, submerge the pipette tip into sample tube and release the bulb to draw sample into pipette. Make sure the lower shaft of the pipette is filled completely and does not have any air pockets. Excess sample volume will go into the lower bulb of the pipette (Image A).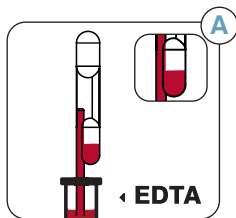

- 3 Squeeze the upper bulb of the pipette to fully dispense sample into the sample port of the device (image B). Excess sample volume will remain in the lower bulb of the pipette. Do not exceed 2 minutes between adding blood and wash solution.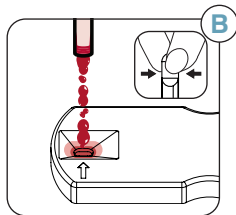

- 4 Grab and twist off the single-use wash solution tip (image C). Hold the single-use wash solution vertically and carefully squeeze to dispense 6 drops into the sample port. The wash solution should be added within 2 minutes of sample addition. **Tip:** If there is pooling of the sample in the sample port after adding wash solution, gently tap the device until liquid flows through the sample port.

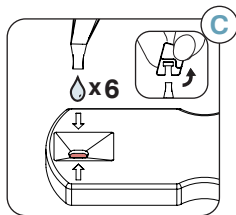- 5 Start a timer and read the device under sufficient lighting at 20 minutes. If both test, T, and control, C, lines appear before 20 minutes, the animal is pregnant, and the test is complete. If there is blood in the result window, allow an additional 5 to 10 minutes for the window to clear.

- 6 If needed, devices can be read up to 50 minutes after the wash solution is added. After that time, results are not valid.

Interpretation of Results


C = Control

T = Test


Look closely and read results in well lit area


- 1 Presence of a red or pink test line, T, indicates that the animal is Pregnant. The test line may range from dark to very light. The presence of a red or pink test line, T, at any intensity is considered Pregnant.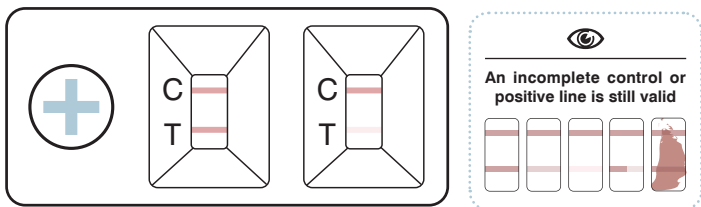

- 2 Absence of test line indicates that the animal is Open.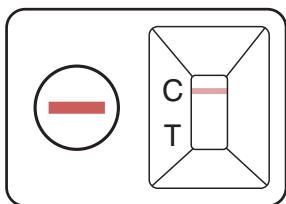

- 3 The test is invalid if the control line, C, does not appear.

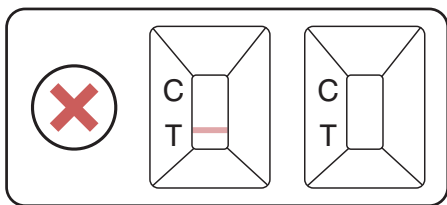Attention Customers

IDEXX encourages producers to work closely with their veterinarian to develop a reproductive management program that is appropriate for their operation.

For technical assistance:

IDEXX USA Tel: +1 800 548 9997 or +1 207 556 4895

IDEXX Europe Tel: +800 727 43399

Contact your IDEXX area manager or distributor or visit our website:

idexx.com/contactlpd

© 2025 IDEXX Laboratories, Inc. All rights reserved.

*IDEXX, Test With Confidence, and Alertys are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

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