

SNAP Feline Triple Test

Test accuracy

Sensitivity and specificity of the SNAP® Feline Triple® Test

Analyte	Reference method	Device result		Total	Sample type	Sensitivity (95% CL)
		+	-			Specificity (95% CL)
Heartworm ¹	+	25	3	238	Serum/plasma	89.3% (71.8%–97%)
	-	1	209			99.5% (97.0%–100%)
FeLV ²	+	97	0	308	Serum/plasma	100% (95.3%–100%)
	-	3	208			98.6% (95.7%–99.7%)
FIV ³	+	100	0	220	Serum/plasma	100% (96.4%–100%)
	-	1	119			99.2% (95.4%–100%)

Reference Method:

1. Necropsy/PetChek® Heartworm PF Antigen Test Kit (5018.02)
2. PetChek® Feline Leukemia Virus Antigen Test Kit (5028.01)
3. FIV Western blot

For retrovirus test interpretation and diagnostic algorithms, please refer to AAFP Feline Retrovirus Management Guidelines, available at catvets.com/guidelines