**SNAP\* 4Dx\* Plus Test**

In vitro diagnostic for the detection of *Dirofilaria immitis* antigen, antibody to *Anaplasma phagocytophylum*, antibody to *Anaplasma platys*, antibody to *Borrelia burgdorferi*, antibody to *Ehrlichia canis*, and antibody to *Ehrlichia ewingii* in canine serum, plasma, or whole blood.

**Precautions and warnings**

- All wastes should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- Male-only heartworm infections typically produce antigen levels that are below the detection capability of this antigen test.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.

**Storage**

- Store at 2-8°C until expiration date.
- SNAP devices and reagents can be stored at room temperature (18-25°C) for 6 months or until the expiration date, whichever occurs first.
- After SNAP devices and reagents are removed from 2-8°C for **more than 24 hours**, the expiration date is 6 months or the printed expiration date, whichever occurs first. If the 6 month expiration date occurs prior to the printed expiration date, record the new date in the space provided on the kit.

Item	Reagents	Quantity
1	1 or 5 bottles Anti- <i>D. immitis</i> /A. <i>phagocytophylum</i> /A. <i>platys</i> /B. <i>burgdorferi</i> /E. <i>canis</i> /E. <i>ewingii</i> ; HRPO conjugate (preserved with ProClin™ 150)	7,0 mL
2	SNAP device	5, 15, 30, or 180
<b>Reagents contained in each device:</b>		
	Substrate solution (preserved with ProClin™ 150)	0,4 mL
	Wash solution	0,6 mL
<b>Other components:</b> transfer pipettes, sample tubes, and reagent rack		

**Conjugate** - H317H412/P261/P280/P333 + P313: May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing vapors. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.

**Substrate Solution** - H317/P261/P280/P302 + P350/P333 + P313: May cause an allergic skin reaction. Avoid breathing vapors. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.

**Wash Solution** - H317H319/H402/H412/P261/P280/P305 + P351 + P338/P337 + P313: Harmful to aquatic life. Harmful to aquatic life with long lasting effects. Avoid breathing vapors. Wear eye protection/face protection. Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

**WARNING**

Caution: This product contains a chemical irritant. Avoid contact with eyes, nose, mouth, and skin. If contact occurs, flush immediately with water for at least 15 minutes. Seek medical attention if irritation persists.


**Sample information**

- Samples must be at room temperature (18-25°C) before beginning the test procedure.
- Serum, plasma, or anti-coagulated whole blood (e.g., EDTA, heparin), either fresh or stored at 2-8°C for up to one week, can be used.
- For longer storage, serum, or plasma can be frozen (-20°C or colder) and then re-centrifuged before use.
- Hemolyzed or lipemic samples will not affect test results.

**Test procedure**

- If stored in a refrigerator, allow all components to equilibrate at room temperature (18-25°C) for 30 minutes before use. **Do not heat.**
- Using the pipette provided, dispense **3 drops of sample** into a new sample tube.
- Holding the bottle vertical, add **4 drops of conjugate** to the sample tube.
- Cap the sample tube and mix it thoroughly by **inverting it 3-5 times**.

- Place the device on a horizontal surface. Add the entire contents of the sample tube to the sample well, being careful not to splash the contents outside of the sample well.
- The sample will flow across the result window, reaching the activation circle in 30-60 seconds. Some sample may remain in the sample well.
- When color **FIRST** appears in the activation circle, push the activator firmly until it is flush with the device body.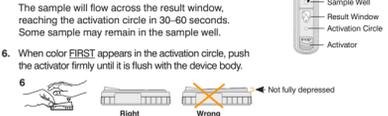

**Note:** Some samples may not flow to the activation circle within 60 seconds, and therefore the circle may not turn color. In this case, press the activator after the sample has flowed across the result window.


7. Read test result at **8 minutes**.

**Note:** The positive control may develop sooner, but results are not complete until 8 minutes.

**Interpreting the test results**

**Positive result**

Any color development in the sample spots indicates the presence of *Dirofilaria immitis* antigen, A. *phagocytophylum* antibody, A. *platys* antibody, B. *burgdorferi* antibody, E. *canis* antibody, or E. *ewingii* antibody in the sample.

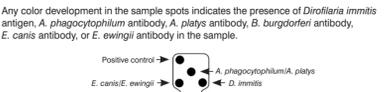**Notes:**

- The A. *phagocytophylum*/A. *platys* spot cannot differentiate between the two species: a positive result indicates presence of antibodies to A. *phagocytophylum* and/or A. *platys*.
- The E. *canis*/E. *ewingii* spot cannot differentiate between the two species: a positive result indicates presence of antibodies to E. *canis* and/or E. *ewingii*.
- In a low percentage of samples (0.027% as reported), interfering substances in the specimen can cause all spots on the device to react as positive. In this event, the sample should be re-assayed as plasma or serum to reduce the likelihood of interference.

**Negative result**

Only positive control spot develops color.

**Invalid results**

- Background**—If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if color background obscures test result, repeat the test.
- No color development**—If the positive control does not develop color, repeat the test.

**Borrelia burgdorferi vaccine cross-reactivity**—The *Borrelia burgdorferi* assay detects antibodies induced as a result of natural infection with the organism and not following immunization with the following vaccines: Recombitek® Lyme, LymeVax®, and Galaxy® Lyme and Nobivac® Lyme. Recombitek is a registered trademark of Merial, Inc. LymeVax is a registered trademark of Pfizer, Inc. Galaxy and Nobivac are registered trademarks of Merck Animal Health.

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**Symbol descriptions**

Use by date	ECREP	Authorized representative in the European Community
Batch code (lot)		
Serial number	TI	Consult instructions for use
Temperature limitation	IVD	In vitro diagnostic
Manufacturer		Date of manufacture
Catalog number	REF	

**IDEXX Technical Support**  
 USA/Canada: 1 800 248 2483 • idexx.com  
 Australia: 1300 44 33 99 • idexx.com.au  
 Europe: idexx.eu

**Assistance technique IDEXX**  
 E.-U./Canada: 1 800 248 2483 • idexx.com  
 Australie: 1300 44 33 99 • idexx.com.au  
 Europe: idexx.eu

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